



**HBV Seroconversion Panel Donor No.1034644**  
**Part Number: HBV9092**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ABBOTT HBV REAL TIME CP/ML  TEST DATE: UNK	DIASORIN HBsAg  TEST DATE: 2/16/2010	ORTHO VITROS ECI HBsAg  TEST DATE: 5/18/2016	ORTHO VITROS ECI ANTI-HBc TOTAL  TEST DATE: 5/18/2016	ORTHO VITROS ECI ANTI-HBc IgM  TEST DATE: 5/18/2016	DIASORIN ANTI-HBc IgM  TEST DATE: 4/1/2010	DIASORIN ANTI-HBc TOTAL  TEST DATE: 3/31/2010
9092-01	12/20/05	NOT DETECTED	3.40	0.05	2.63	0.07	0.16	3.88
9092-02	12/22/05	NOT DETECTED	0.20	0.27	2.88	NOT TESTED	0.16	2.76
9092-03	12/28/05	NOT DETECTED	0.02	0.05	2.67	0.06	0.14	4.07
9092-04	01/03/06	NOT DETECTED	0.08	0.07	2.95	0.07	0.16	5.68
9092-05	01/05/06	NOT DETECTED	0.20	0.06	3.44	0.05	0.16	5.08
9092-06	01/12/06	NOT DETECTED	0.06	4.65	2.74	NOT TESTED	0.16	4.36
9092-07	01/17/06	NOT DETECTED	0.20	0.06	2.93	NOT TESTED	0.20	3.36
9092-08	01/19/06	<52	0.05	0.06	2.94	0.06	0.14	3.47
9092-09	01/24/06	113	0.05	0.07	2.86	0.06	0.09	3.93
9092-10	01/26/06	152	0.17	0.07	3.2	0.04	0.09	4.00
9092-11	01/31/06	665	0.40	0.37	3.08	0.05	0.20	2.64
9092-12	02/02/06	877	1.00	0.79	3.19	0.03	0.14	4.53
9092-13	02/07/06	9611	7.00	8.33	3.00	0.03	0.26	1.76
9092-14	02/09/06	44953	12.55	16.5	3.19	0.05	0.14	4.27
9092-15	02/22/06	2902381	72.33	1660	2.98	0.05	0.14	3.73
9092-16	02/27/06	29699858	60.60	6430	3.18	0.05	0.16	2.80
9092-17	03/06/06	1789593439	60.00	4000	2.05	0.06	0.16	2.64
9092-18	03/08/06	>3410000000	71.93	2180	0.76	0.11	0.23	1.07
9092-19	03/15/06	>3410000000	60.40	728	0.19	0.6	0.68	0.36
9092-20	03/22/06	>3410000000	60.20	522	0.06	2.21	3.64	0.28
9092-21	03/29/06	1006362346	60.20	397	0.04	3.29	8.28	0.20
9092-22	04/05/06	283631446	60.20	668	0.06	3.44	9.84	0.24
9092-23	04/19/06	7460267	60.40	1900	0.04	3.74	11.24	0.12
9092-24	05/01/06	1238	59.80	6380	0.03	NOT TESTED	10.60	0.12
9092-25	05/03/06	1078	60.20	6280	0.04	NOT TESTED	7.76	0.12
9092-26	05/10/06	729	60.20	5070	0.06	NOT TESTED	6.24	0.12
9092-27	05/17/06	1078	61.00	2560	0.11	3.2	5.80	0.12
9092-28	05/31/06	781	61.20	488	0.11	NOT TESTED	4.72	0.12
9092-29	06/06/06	205	41.00	110	0.15	NOT TESTED	3.92	0.08
9092-30	06/08/06	471	40.60	73.6	0.11	2.25	5.60	0.08
9092-31	06/13/06	265	9.60	13.2	0.19	1.63	2.56	0.08
9092-32	06/15/06	210	7.00	8.76	0.13	1.89	2.88	0.08
9092-33	06/20/06	96	10.60	1.97	0.16	1.4	2.28	0.08
9092-34	06/22/06	82	2.00	0.81	0.18	1.18	1.44	0.08
9092-35	06/27/06	65	1.00	0.23	0.17	1.01	1.20	0.08
9092-36	06/29/06	259	0.80	0.21	0.15	1.00	0.96	0.12
9092-37	07/06/06	58	0.40	0.09	0.13	1.00	1.08	0.08

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)



**HBV Seroconversion Panel Donor No.1034644  
Part Number: HBV9092**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO VITROS ECi HBeAg	ORTHO VITROS ECi ANTI-HBe	ORTHO VITROS ECi ANTI-HBs QUANT	DIASORIN ANTI-HBs QUANT
	TEST DATE: 5/18/2016	TEST DATE: 5/18/2016	TEST DATE: 5/18/2016	TEST DATE: UNK
9092-01	0.22	0.18	2.1	0.31
9092-02	0.7	0.17	NOT TESTED	0.27
9092-03	0.24	NOT TESTED	0.7	0.24
9092-04	0.28	NOT TESTED	1.3	0.20
9092-05	0.23	0.29	1.1	0.21
9092-06	QNS	NOT TESTED	NOT TESTED	0.18
9092-07	0.22	NOT TESTED	0.6	0.16
9092-08	0.21	NOT TESTED	0.3	0.15
9092-09	0.24	NOT TESTED	0.6	0.16
9092-10	0.23	0.16	0.5	0.16
9092-11	0.24	0.17	0.5	0.20
9092-12	0.23	0.16	0.7	0.10
9092-13	0.28	0.16	1.4	0.14
9092-14	0.33	0.16	0.3	0.14
9092-15	73	0.05	0.3	0.21
9092-16	831	0.01	0.9	0.16
9092-17	3840	0.00	0.9	0.23
9092-18	3560	0.00	1.6	0.24
9092-19	3260	0.00	1.3	0.22
9092-20	3270	NOT TESTED	1.7	0.20
9092-21	2690	NOT TESTED	1.5	0.20
9092-22	2800	NOT TESTED	1.6	0.17
9092-23	2670	NOT TESTED	2.5	0.22
9092-24	179	NOT TESTED	1.3	0.20
9092-25	107	NOT TESTED	NOT TESTED	0.25
9092-26	14.2	NOT TESTED	NOT TESTED	0.27
9092-27	18.5	NOT TESTED	2.7	0.30
9092-28	5.17	NOT TESTED	NOT TESTED	0.37
9092-29	4.24	NOT TESTED	1.7	0.27
9092-30	2.76	0.38	1.3	0.31
9092-31	1.77	NOT TESTED	2.1	0.20
9092-32	1.09	NOT TESTED	0.8	0.30
9092-33	0.92	NOT TESTED	0.8	0.26
9092-34	0.77	0.77	0.9	0.25
9092-35	0.6	0.6	0.1	0.29
9092-36	0.57	0.57	3.1	0.22
9092-37	0.44	0.44	1.3	0.21

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)