



**HCV Seroconversion Panel Donor No. 69263
Part Number: HCV10008**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

| PANEL MEMBER | DATE OF DRAW | ALT Normal Range (10-45 u/L) | ALPHA HCV by PCR | ABBOTT MUREX ANTI-HCV 4.0 | ABBOTT MUREX HCV Ag/Ab Combo | ABBOTT ANTI-HCV Lab # 1 | ABBOTT ANTI-HCV 3.0 Lab # 5 | ABBOTT PRISM HCV Lab # 5 |
|--------------|--------------|------------------------------|------------------|---------------------------|------------------------------|-------------------------|-----------------------------|--------------------------|
| | | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: 9/21/2000 | TEST DATE: 10/4/2000 |
| 10008-01 | 10/27/98 | 5 | NEGATIVE | 0.097 | 0.286 | 0.111 | 0.080 | 0.070 |
| 10008-02 | 10/29/98 | 5 | NEGATIVE | 0.086 | 0.310 | 0.089 | 0.070 | 0.060 |
| 10008-03 | 11/04/98 | 9 | NEGATIVE | 0.095 | 0.328 | 0.111 | 0.080 | 0.060 |
| 10008-04 | 11/06/98 | 8 | POSITIVE | 0.092 | 0.324 | 0.111 | 0.070 | 0.060 |
| 10008-05 | 11/11/98 | 6 | POSITIVE | 0.086 | 7.279 | 0.111 | 0.070 | 0.060 |
| 10008-06 | 11/13/98 | 11 | POSITIVE | 0.095 | 5.552 | 0.111 | 0.070 | 0.060 |
| 10008-07 | 11/18/98 | 14 | POSITIVE | 0.094 | 9.800 | 0.111 | 0.070 | 0.050 |
| 10008-08 | 11/20/98 | 12 | POSITIVE | 0.083 | 9.831 | 0.111 | 0.070 | 0.060 |
| 10008-09 | 11/25/98 | 22 | POSITIVE | 0.091 | 11.121 | 0.111 | 0.070 | 0.070 |
| 10008-10 | 11/27/98 | 19 | POSITIVE | 0.086 | 11.248 | 0.089 | 0.060 | 0.060 |

| PANEL MEMBER | ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack Lab # 5 | ABBOTT Architect ANTI-HCV | ABBOTT PRISM ANTI-HCV | ROCHE Cobas Core ANTI-HCV EIA Lab # 5 | ORTHO Enhanced SAve ANTI-HCV 3.0 Lab # 5 | ORTHO ECi HBsAg | ORTHO ECi ANTI-HCV | ORTHO ECi ANTI-HIV 1/2 |
|--------------|--|---------------------------|-----------------------|---------------------------------------|--|-----------------------|-----------------------|------------------------|
| | TEST DATE: 7/20/2000 | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: 9/6/2000 | TEST DATE: 9/18/2000 | TEST DATE: 11/16/2018 | TEST DATE: 11/16/2018 | TEST DATE: 11/16/2018 |
| 10008-01 | 0.110 | 0.100 | 0.135 | 0.060 | 0.000 | 0.06 | 0.01 | 0.12 |
| 10008-02 | 0.080 | 0.090 | 0.095 | 0.050 | 0.000 | 0.05 | 0.00 | 0.11 |
| 10008-03 | 0.080 | 0.200 | 0.112 | 0.060 | 0.000 | 0.05 | 0.01 | 0.12 |
| 10008-04 | 0.090 | 0.090 | 0.114 | 0.060 | 0.000 | 0.04 | 0.01 | 0.12 |
| 10008-05 | 0.090 | 0.060 | 0.105 | 0.060 | 0.000 | 0.04 | 0.00 | 0.13 |
| 10008-06 | 0.170 | 0.090 | 0.121 | 0.050 | 0.000 | 0.05 | 0.01 | 0.11 |
| 10008-07 | 0.140 | 0.090 | 0.118 | 0.060 | 0.010 | 0.05 | 0.01 | 0.10 |
| 10008-08 | 0.110 | 0.080 | 0.119 | 0.060 | 0.000 | 0.04 | 0.01 | 0.12 |
| 10008-09 | 0.090 | 0.100 | 0.127 | 0.060 | 0.000 | 0.05 | 0.01 | 0.11 |
| 10008-10 | 0.110 | 0.080 | 0.117 | 0.060 | 0.000 | 0.04 | 0.01 | 0.11 |

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com