



**HCV Seroconversion Panel Donor No. 68896  
Part Number: HCV10009**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

**HCV Genotype: 1a/1b**

| PANEL MEMBER | DATE OF DRAW | ALT Normal Range (10-45 u/L)<br>TEST DATE: UNK | ALPHA HCV by PCR<br>TEST DATE: UNK | ABBOTT MUREX ANTI-HCV 4.0<br>TEST DATE: UNK | ABBOTT MUREX HCV Ag/Ab Combo<br>TEST DATE: UNK | ABBOTT ANTI-HCV Lab # 1<br>TEST DATE: 9/2/1999 | ABBOTT ANTI-HCV 3.0 Lab # 5<br>TEST DATE: 9/21/2000 | ABBOTT PRISM HCV Lab # 5<br>TEST DATE: 10/4/2000 |
|--------------|--------------|--|------------------------------------|---|--|--|---|--|
| 10009-01     | 09/23/98     | 11   | NEGATIVE                           | 0.297                                       | 0.313  | 0.200  | 0.130   | 0.090  |
| 10009-02     | 09/25/98     | 23   | NEGATIVE                           | 0.263                                       | 0.330  | 0.178  | 0.110   | 0.090  |
| 10009-03     | 10/01/98     | 47   | NEGATIVE                           | 0.201                                       | 0.337  | 0.156  | 0.090   | 0.110  |
| 10009-04     | 10/03/98     | 44   | POSITIVE                           | 0.224                                       | 0.367  | 0.133  | 0.110   | 0.080  |
| 10009-05     | 10/08/98     | 34   | POSITIVE                           | 0.215                                       | 0.398  | 0.178  | 0.090   | 0.080  |
| 10009-06     | 10/10/98     | 23   | POSITIVE                           | 0.188                                       | 0.354  | 0.156  | 0.110   | 0.080  |

| PANEL MEMBER | ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack Lab # 5<br>TEST DATE: 7/20/2000 | ABBOTT Architect ANTI-HCV<br>TEST DATE: UNK | ABBOTT PRISM ANTI-HCV<br>TEST DATE: UNK | ROCHE Cobas Core ANTI-HCV EIA<br>TEST DATE: 9/6/2000 | ORTHO Enhanced SAVE ANTI-HCV 3.0 Lab # 5<br>TEST DATE: 9/18/2000 | ORTHO Eci HBsAg<br>TEST DATE: 11/16/2018 | ORTHO Eci ANTI-HCV<br>TEST DATE: 11/16/2018 | ORTHO Eci ANTI-HIV ½<br>TEST DATE: 11/16/2018 |
|--------------|--|---|---|--|--|--|---|---|
| 10009-01     | 0.170  | 0.080                                       | 0.120                                   | 0.270  | 0.000  | 0.07                                     | 0.02  | 0.12  |
| 10009-02     | 0.170  | 0.060                                       | 0.108                                   | 0.150  | 0.000  | 0.07                                     | 0.01  | 0.12  |
| 10009-03     | 0.110  | 0.050                                       | 0.102                                   | 0.110  | 0.000  | 0.05                                     | 0.01  | 0.13  |
| 10009-04     | 0.150  | 0.050                                       | 0.098                                   | 0.100  | 0.000  | 0.05                                     | 0.01  | 0.11  |
| 10009-05     | 0.160  | 0.060                                       | 0.116                                   | 0.090  | 0.000  | 0.06                                     | 0.01  | 0.12  |
| 10009-06     | 0.150  | 0.050                                       | 0.134                                   | 0.100  | 0.000  | 0.05                                     | 0.01  | 0.14  |

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)