



**HCV Seroconversion Panel Donor No. 67391  
Part Number: HCV10026**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

**HCV Genotype: 3a**

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ALPHA HCV by PCR	ABBOTT MUREX ANTI-HCV 4 0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack Lab # 5
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/25/1999	TEST DATE: UNK
10026-01	03/13/98	7	NEGATIVE	0.114	0.302	0.135	0.110
10026-02	03/15/98	11	NEGATIVE	0.105	0.322	0.135	0.100
10026-03	03/20/98	10	NEGATIVE	0.130	0.322	0.135	0.180
10026-04	03/22/98	5	NEGATIVE	0.120	0.332	0.135	0.110
10026-05	04/10/98	0	NEGATIVE	0.155	0.349	0.162	0.160
10026-06	05/04/98	13	POSITIVE	0.138	0.356	0.108	0.230
10026-07	05/09/98	17	POSITIVE	0.141	4.329	0.162	0.130
10026-08	05/11/98	10	POSITIVE	0.141	4.064	0.135	0.100
10026-09	05/16/98	31	POSITIVE	0.120	2.047	0.135	0.100
10026-10	05/25/98	120	POSITIVE	0.124	3.064	0.162	0.130
10026-11	05/29/98	324	POSITIVE	0.135	1.264	0.135	0.190
10026-12	06/01/98	356	POSITIVE	0.208	0.976	0.189	1.370

PANEL MEMBER	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV	SANOFI Monolisa ANTI-HCV PLUS Lab # 5	ORTHO ECi HBsAg	ORTHO ECi ANTI-HCV	ORTHO ECi ANTI-HIV ½
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/20/2000	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10026-01	0.060	0.166	0.170	0.05	0.01	0.14
10026-02	0.050	0.101	0.170	0.05	0.01	0.12
10026-03	0.060	0.120	0.170	0.06	0.01	0.13
10026-04	0.050	0.117	0.170	0.05	0.01	0.13
10026-05	0.070	0.102	0.180	0.07	0.02	0.12
10026-06	0.100	0.133	0.180	0.06	0.01	0.12
10026-07	0.080	0.118	0.190	0.06	0.02	0.13
10026-08	0.060	0.091	0.180	0.06	0.01	0.13
10026-09	0.060	0.174	0.190	0.07	0.02	0.12
10026-10	0.070	0.149	0.200	0.06	0.01	0.14
10026-11	0.100	0.121	0.210	0.06	0.01	0.12
10026-12	0.590	0.547	0.200	0.07	0.07	0.12

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)