



**HCV Seroconversion Panel Donor No. 67539  
Part Number: HCV10029**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

**HCV Genotype: 1**

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ALPHA HCV by PCR	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1	ABBOTT ANTI-HCV 3.0 Lab # 5
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/27/1999	TEST DATE: 10/16/2000
10029-01	04/20/98	3	NEGATIVE	0.093	0.332	0.131	0.140
10029-02	04/25/98	7	NEGATIVE	0.090	0.325	0.131	0.100
10029-03	04/28/98	4	NEGATIVE	0.092	0.308	0.148	0.060
10029-04	05/13/98	4	NEGATIVE	0.098	0.315	0.115	0.070
10029-05	05/17/98	7	NEGATIVE	0.099	0.322	0.131	0.070
10029-06	05/23/98	6	NEGATIVE	0.095	0.346	0.131	0.090
10029-07	05/25/98	7	NEGATIVE	0.101	0.325	0.098	0.050
10029-08	05/30/98	8	POSITIVE	0.101	0.336	0.115	0.030
10029-09	06/07/98	16	POSITIVE	0.113	7.541	0.148	0.000
10029-10	06/09/98	13	POSITIVE	0.102	7.007	0.115	0.080
10029-11	06/14/98	20	POSITIVE	0.098	9.747	0.131	0.150
10029-12	06/16/98	17	POSITIVE	0.098	8.610	0.131	0.050
10029-13	06/21/98	18	POSITIVE	0.108	10.812	0.115	0.070
10029-14	06/23/98	13	POSITIVE	0.117	9.777	0.098	0.060
10029-15	06/28/98	19	POSITIVE	0.101	9.729	0.115	0.060
10029-16	06/30/98	24	POSITIVE	0.095	10.401	0.115	0.060

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)



**HCV Seroconversion Panel Donor No. 67539  
Part Number: HCV10029**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT IMx HCV 3.0 Reagent Pack Lab # 5 TEST DATE: 10/17/2000	ABBOTT Architect ANTI-HCV TEST DATE: UNK	ABBOTT Prism ANTI-HCV TEST DATE: UNK	ORTHO Enhanced SAVe ANTI-HCV 3.0 Lab # 5 TEST DATE: 9/29/2000	ORTHO ECi HBsAg TEST DATE: 11/16/2018	ORTHO ECi ANTI-HCV TEST DATE: 11/16/2018	ORTHO ECi ANTI-HIV 1/2 TEST DATE: 11/16/2018
10029-01	0.070	0.040	0.110	0.000	0.04	0.01	0.11
10029-02	0.120	0.040	0.093	0.000	0.05	0.00	0.11
10029-03	0.090	0.040	0.094	0.000	0.05	0.00	0.11
10029-04	0.130	0.040	0.100	0.000	0.05	0.01	0.11
10029-05	0.130	0.040	0.108	0.000	0.05	0.01	0.11
10029-06	0.140	0.040	0.103	0.000	0.05	0.01	0.12
10029-07	0.110	0.040	0.115	0.000	0.05	0.01	0.12
10029-08	0.230	0.040	0.128	0.000	0.05	0.01	0.11
10029-09	0.110	0.050	0.124	0.000	0.05	0.01	0.11
10029-10	0.150	0.040	0.111	0.000	0.05	0.01	0.11
10029-11	0.120	0.040	0.110	0.000	0.05	0.01	0.10
10029-12	0.090	0.040	0.115	0.000	0.05	0.01	0.11
10029-13	0.110	0.040	0.118	0.000	0.05	0.01	0.11
10029-14	0.110	0.050	0.108	0.000	0.05	0.01	0.11
10029-15	0.120	0.040	0.104	0.000	0.05	0.01	0.12
10029-16	0.130	0.040	0.099	0.000	0.05	0.01	0.11

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)