



**HCV Seroconversion Panel Donor No.66113
Part Number: HCV10032**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

HCV Genotype: 3a

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ALPHA HCV by PCR	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1	ABBOTT Architect ANTI-HCV
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/25/1999	TEST DATE: UNK
10032-01	12/24/97	2	NEGATIVE	0.086	0.312	0.162	0.040
10032-02	12/26/97	6	NEGATIVE	0.090	0.315	0.189	0.050
10032-03	01/02/97	4	NEGATIVE	0.090	0.305	0.162	0.050
10032-04	01/04/98	9	POSITIVE	0.089	0.315	0.135	0.040
10032-05	01/09/98	11	POSITIVE	0.098	0.705	0.162	0.050
10032-06	01/11/98	12	POSITIVE	0.105	0.346	0.162	0.050
10032-07	01/16/98	9	POSITIVE	0.099	0.325	0.189	0.050
10032-08	01/18/98	6	POSITIVE	0.093	0.332	0.162	0.050
10032-09	01/23/98	6	POSITIVE	0.090	0.342	0.162	0.050
10032-10	01/25/98	11	POSITIVE	0.087	0.366	0.135	0.040
10032-11	01/30/98	15	POSITIVE	0.092	0.349	0.162	0.050
10032-12	02/01/98	13	NOT TESTED	0.090	0.384	0.189	0.050
10032-13	02/06/98	27	POSITIVE	0.092	0.373	0.189	0.040
10032-14	02/08/98	40	POSITIVE	0.105	0.373	0.162	0.040
10032-15	02/13/98	51	POSITIVE	0.098	0.363	0.162	0.050
10032-16	02/15/98	52	POSITIVE	0.101	0.356	0.189	0.050

PANEL MEMBER	ABBOTT PRISM ANTI-HCV	ORTHO HCV 3.0 ELISA Testsystem Enhanced SAve	ROCHE Cobas Core ANTI-HCV EIA	ORTHO ECi HBsAg	ORTHO ECi ANTI-HCV	ORTHO ECi ANTI-HIV ½
	TEST DATE: UNK	TEST DATE: 10/23/2000	TEST DATE: 9/7/2000	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10032-01	0.134	0.010	0.100	0.05	0.01	0.08
10032-02	0.086	0.000	0.100	0.05	0.01	0.08
10032-03	0.097	0.010	0.100	0.05	0.01	0.08
10032-04	0.084	0.010	0.100	0.05	0.01	0.09
10032-05	0.099	0.010	0.100	0.06	0.01	0.08
10032-06	0.089	0.010	0.100	0.05	0.01	0.08
10032-07	0.100	0.000	0.100	0.05	0.01	0.07
10032-08	0.094	0.010	0.100	0.05	0.01	0.07
10032-09	0.087	0.010	0.100	0.05	0.01	0.08
10032-10	0.069	0.010	0.100	0.05	0.01	0.08
10032-11	0.104	0.010	0.100	0.05	0.01	0.09
10032-12	0.093	0.010	0.100	0.04	0.01	0.09
10032-13	0.097	0.000	0.100	0.05	0.01	0.09
10032-14	0.083	0.000	0.100	0.06	0.01	0.10
10032-15	0.091	0.000	0.100	0.06	0.01	0.10
10032-16	0.093	0.000	0.100	0.07	0.02	0.09

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com