



**HCV Seroconversion Panel Donor No.420299823
Part Number: HCV10058**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

HCV Genotype: 3a

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	VENDOR ANTI-HCV Ab	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	CHIRON PCR	ABBOTT HCV Ab	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 11/23/1999	TEST DATE: UNK	TEST DATE: UNK
10058-01	05/30/97	11	NEGATIVE	5.240	10.794	4.682	0.413	10.680	11.327
10058-02	06/03/97	18	NEGATIVE	5.271	11.380	1.623	0.457	12.890	11.328
10058-03	06/06/97	14	NEGATIVE	5.284	11.000	1.613	0.674	13.390	11.430
10058-04	06/10/97	20	POSITIVE	5.323	11.094	5.194	0.783	13.460	11.591
10058-05	06/13/97	8	POSITIVE	5.293	11.286	0.365	0.978	13.790	11.742
10058-06	06/17/97	17	POSITIVE	5.315	11.686	0.463	1.130	11.990	12.885
10058-07	06/20/97	29	POSITIVE	5.359	11.965	1.812	1.217	13.130	12.830

PANEL MEMBER	ORTHO HCV Ab	ORTHO RIBA C100	ORTHO RIBARIBA C33	ORTHO RIBA C22	ORTHO RIBA NS5	ORTHO RIBA SOD	ORTHO ECi HBsAg	ORTHO ECi ANTI-HCV	ORTHO ECi ANTI-HIV 1/2
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10058-01	5.024	2	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	0.04	9.35	0.09
10058-02	5.024	2	2	NEGATIVE	NEGATIVE	NEGATIVE	0.05	11.3	0.08
10058-03	5.024	1	2	NEGATIVE	NEGATIVE	NEGATIVE	0.05	12.2	0.07
10058-04	5.024	1	2	POSITIVE	NEGATIVE	NEGATIVE	0.05	12.7	0.10
10058-05	5.024	2	3	POSITIVE	NEGATIVE	NEGATIVE	0.05	14.0	0.08
10058-06	5.024	2	3	POSITIVE	NEGATIVE	NEGATIVE	0.06	14.4	0.08
10058-07	5.024	2	3	NEGATIVE	NEGATIVE	NEGATIVE	0.05	14.2	0.07

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com