



HCV Seroconversion Panel Donor No. 60996
Part Number: HCV6215

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L) TEST DATE: UNK	ABBOTT MUREX ANTI-HCV 4.0 TEST DATE: UNK	ABBOTT MUREX HCV Ag/Ab Combo TEST DATE: UNK	ABBOTT IMx ANTI-HCV (Lab #1) TEST DATE: 1/13/1997	ABBOTT IMx ANTI-HCV (Lab#2) TEST DATE: 9/23/1996	ABBOTT ANTI-HCV 2.0 TEST DATE: 9/6/1996	ABBOTT Architect ANTI-HCV TEST DATE: UNK
6215-01	01/19/96	25	0.097	10.943	0.47	0.443	0.140	0.070
6215-02	01/22/96	20	0.102	9.923	0.51	0.441	0.155	0.060
6215-03	01/29/96	40	0.305	8.763	0.69	0.574	0.275	0.110
6215-04	02/08/96	160	5.262	11.462	0.52	1.831	3.518	3.610

PANEL MEMBER	ABBOTT PRISM ANTI-HCV TEST DATE: UNK	ABBOTT PRISM ANTI-HCV TEST DATE: 12/10/1996	ABBOTT EIA ANTI-HCV 2 nd Generation TEST DATE: 1/15/1997	MUREX ANTI-HCV Version III TEST DATE: 12/17/1996	ABBOTT AxSYM ANTI-HCV TEST DATE: 12/11/1996	ROCHE Cobas Core ANTI-HCV EIA TEST DATE: 12/16/1996	SORIN ANTI-HCV ETI-AB- HCVK-3 TEST DATE: 12/16/1996	ORTHO RIBA 3.0 C100 TEST DATE: UNK
6215-01	0.150	0.04	0.11	0.09	0.41	0.12	0.03	-
6215-02	0.170	0.04	0.11	0.09	0.48	0.09	0.02	-
6215-03	1.320	0.15	0.24	0.12	0.40	0.36	0.26	-
6215-04	0.190	1.47	3.61	2.82	0.57	>5.0	3.95	-

PANEL MEMBER	ORTHO RIBA 3.0 C33 TEST DATE: UNK	ORTHO RIBA 3.0 C22 TEST DATE: UNK	ORTHO RIBA 3.0 NS5 TEST DATE: UNK	ORTHO RIBA 3.0 SOD TEST DATE: UNK	ORTHO RIBA 3.0 RESULT TEST DATE: UNK	CHIRON RIBA ANTI-HCV 3.0 SIA TEST DATE: 12/11/1996	CHIRON HCV RNA (bDNA) Copies/ML TEST DATE: 12/17/1996
6215-01	-	-	-	-	-	NEGATIVE	>12000000
6215-02	-	-	-	-	-	NEGATIVE	104100000
6215-03	-	-	-	-	-	NEGATIVE	58140000
6215-04	+/-	4+	-	-	INDETERMINATE	POSITIVE	48140000

PANEL MEMBER	ORTHO ANTI-HCV 3.0 (SAVE) TEST DATE: 8/15/1996	ORTHO ANTI-HCV 3.0 ELISA TEST DATE: 1/21/1997	ORTHO ANTI-HCV 3.0 TEST DATE: UNK	MONALISA ANTI-HCV New Antigens TEST DATE: 9/25/1996	MONALISA ANTI-HCV TEST DATE: 12/16/1996	PCR RNA Qualitative Result (Lab #3) TEST DATE: UNK	PCR RNA Quantitative Result (Lab #4) TEST DATE: UNK
6215-01	0.003	0.01	0.003	0.034	0.07	POSITIVE	>5000000
6215-02	0.012	0.01	0.000	0.031	0.06	POSITIVE	>5000000
6215-03	0.018	0.01	0.005	0.031	0.11	POSITIVE	>5000000
6215-04	4.664	3.05	1.154	3.696	3.2	POSITIVE	>5000000

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com