



HCV Seroconversion Panel Donor No. 60779
Part Number: HCV6216

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L) TEST DATE: UNK	ABBOTT MUREX ANTI-HCV 4 0 TEST DATE: UNK	ABBOTT MUREX HCV Ag/Ab Combo TEST DATE: UNK	ABBOTT IMx ANTI-HCV TEST DATE: 9/23/1996	ABBOTT ANTI- HCV 2.0 TEST DATE: 9/6/1996	ABBOTT Architect ANTI-HCV TEST DATE: UNK	ABBOTT PRISM ANTI-HCV TEST DATE: UNK
6216-01	11/27/95	6	0.126	0.311	0.508	0.234	0.120	0.120
6216-02	11/30/95	6	0.127	0.308	0.498	0.202	0.120	0.130
6216-03	12/05/95	5	0.129	0.328	0.462	0.254	0.100	0.110
6216-04	12/07/95	5	0.126	0.331	0.462	0.249	0.100	0.150
6216-05	12/12/95	4	0.150	0.338	0.475	0.243	0.110	0.210
6216-06	12/14/95	5	0.144	0.358	0.462	0.234	0.100	0.140
6216-07	12/20/95	4	0.197	0.368	0.533	1.380	2.930	1.670

PANEL MEMBER	ABBOTT PRISM ANTI-HCV TEST DATE: 12/10/1996	ABBOTT EIA ANTI-HCV 2 nd Generation TEST DATE: 1/15/1997	ORTHO ANTI-HCV 3.0 (SAVe) TEST DATE: 8/15/1996	ORTHO ANTI-HCV 3.0 TEST DATE: UNK	MUREX ANTI-HCV Version III TEST DATE: 12/17/1996	ABBOTT AxSYM ANTI-HCV TEST DATE: 12/11/1996	ROCHE Cobas Core ANTI-HCV EIA TEST DATE: 12/16/1996	SORIN ANTI-HCV ETI-AB- HCVK-3 TEST DATE: 12/16/1996
6216-01	0.07	0.20	0.039	0.005	0.11	0.46	0.26	0.11
6216-02	0.07	0.24	0.051	0.008	0.11	0.48	0.24	0.11
6216-03	0.06	0.19	0.045	0.003	0.11	0.48	0.27	0.12
6216-04	0.06	0.21	0.039	0.005	0.11	0.50	0.24	0.08
6216-05	0.07	0.20	0.045	0.006	0.11	0.50	0.32	0.11
6216-06	0.07	0.21	0.036	0.005	0.11	0.54	0.21	0.09
6216-07	1.13	2.20	3.531	1.719	0.11	0.55	5.16	3.37

PANEL MEMBER	SANOFI Monolisa ANTI-HCV New Antigens TEST DATE: 9/25/1996	SANOFI Monolisa Anti-HCV New Antigens TEST DATE: 12/16/1996	ORTHO RIBA 3.0 C100 TEST DATE: UNK	ORTHO RIBA 3.0 C33 TEST DATE: UNK	ORTHO RIBA 3.0 C22 TEST DATE: UNK	ORTHO RIBA 3.0 NS5 TEST DATE: UNK	ORTHO RIBA 3.0 SOD TEST DATE: UNK
6216-01	0.076	0.11	-	-	-	-	-
6216-02	0.087	0.11	-	-	-	-	-
6216-03	0.061	0.11	-	-	-	-	-
6216-04	0.044	0.11	-	-	-	-	-
6216-05	0.084	0.10	-	-	-	-	-
6216-06	0.073	0.11	-	-	-	-	-
6216-07	0.155	0.30	-	-	3+	-	-

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com



HCV Seroconversion Panel Donor No. 60779
Part Number: HCV6216

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	CHIRON RIBA ANTI-HCV 3.0 SIA TEST DATE: 12/11/1996	CHIRON HCV RNA (bDNA) Copies/MI TEST DATE: 12/17/1996	PCR RNA Qualitative Result (Lab # 3) TEST DATE: UNK	PCR RNA Quantitative Result (Lab # 4) TEST DATE: UNK	ORTHO Eci HBsAg TEST DATE: 11/16/2018	ORTHO Eci ANTI-HCV TEST DATE: 11/16/2018	ORTHO Eci ANTI-HIV ½ TEST DATE: 11/16/2018
6216-01	NEGATIVE	<200,000	NEGATIVE	<100	0.06	0.07	0.17
6216-02	NEGATIVE	<200,000	NEGATIVE	<100	0.05	0.08	0.18
6216-03	NEGATIVE	<200,000	NEGATIVE	<100	0.05	0.06	0.18
6216-04	NEGATIVE	<200,000	NEGATIVE	<100	0.05	0.06	0.17
6216-05	NEGATIVE	<200,000	NEGATIVE	<100	0.05	0.08	0.18
6216-06	NEGATIVE	<200,000	NEGATIVE	<100	0.05	0.06	0.17
6216-07	C22	<200,000	NEGATIVE	500	0.06	7.68	0.20

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com