



**HCV Seroconversion Panel Donor No. 62507**  
**Part Number: HCV6223**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT IMx ANTI-HBc	ABBOTT PRISM ANTI-HCV	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 6/18/1997	TEST DATE: 6/18/1997	TEST DATE: UNK	TEST DATE: UNK
6223-01	08/12/96	7	0.10	0.321	0.72	0.11	0.20	0.167
6223-02	08/15/96	1	0.11	0.331	0.53	0.13	0.21	0.153
6223-03	09/04/96	7	0.10	0.324	0.55	0.11	0.21	0.142
6223-04	09/06/96	3	0.11	0.351	0.57	0.11	0.18	0.162
6223-05	09/11/96	8	0.12	0.334	0.53	0.11	0.18	0.162
6223-06	09/13/96	5	0.10	0.314	0.64	0.11	0.19	0.191
6223-07	09/18/96	9	0.09	0.321	0.59	0.11	0.19	0.194
6223-08	09/20/96	7	0.09	0.307	0.65	0.11	0.19	0.154
6223-09	09/25/96	5	0.10	0.311	0.59	0.11	0.19	0.181
6223-10	09/27/96	5	0.10	0.307	0.49	0.12	0.18	0.153
6223-11	10/02/96	7	0.11	0.324	0.56	0.12	0.20	0.170
6223-12	10/04/96	8	0.11	0.334	0.50	0.12	0.19	0.165
6223-13	10/09/96	6	0.12	0.318	0.55	0.12	0.21	0.179
6223-14	10/11/96	7	0.12	0.304	0.55	0.12	0.16	0.173
6223-15	10/16/96	6	0.10	0.301	0.53	0.16	0.28	0.251
6223-16	10/18/96	6	0.10	0.304	0.51	0.34	0.62	0.506
6223-17	10/22/96	7	0.11	0.301	0.50	0.79	2.19	1.505
6223-18	10/25/96	6	0.12	0.318	0.53	1.20	3.10	1.757
6223-19	10/30/96	5	0.12	0.341	0.50	1.19	4.45	1.916
6223-20	11/03/96	6	0.14	0.334	0.51	1.16	4.06	1.898
6223-21	11/07/96	5	0.13	0.314	0.61	1.17	3.69	1.777
6223-22	11/10/96	5	0.11	0.324	0.54	1.16	3.25	1.314

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)



**HCV Seroconversion Panel Donor No. 62507**  
**Part Number: HCV6223**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT ANTI-HCV 2.0 Lab # 1	ABBOTT ANTI-HCV 2.0 Lab # 2	ABBOTT AxSYM ANTI-HCV	ORTHO ANTI-HCV 3.0 (SAVe)	ORTHO ELISA ANTI-HCV 3.0	BOEHRINGER MANNHEIM Enzymuntest ANTI-HCV	ORTHO RIBA 3.0 C100	ORTHO RIBA 3.0 C33
	TEST DATE: 8/14/1997	TEST DATE: 6/25/1997	TEST DATE: 6/18/1997	TEST DATE: UNK	TEST DATE: 6/18/1997	TEST DATE: 6/5/1997	TEST DATE: 9/25/1997	TEST DATE: 9/25/1997
6223-01	0.30	0.23	0.59	0.008	0.04	0.03	-	-
6223-02	0.27	0.23	0.54	0.007	0.05	0.05	-	-
6223-03	0.30	0.23	0.61	0.010	0.06	0.06	-	-
6223-04	0.30	0.25	0.60	0.005	0.03	0.05	-	+/-
6223-05	0.32	0.23	0.52	0.007	0.03	0.05	-	+/-
6223-06	0.25	0.22	0.58	0.012	0.03	0.04	-	+/-
6223-07	0.25	0.24	0.59	0.007	0.04	0.05	-	+/-
6223-08	0.28	0.25	0.58	0.010	0.02	0.05	-	+/-
6223-09	0.34	0.24	0.60	0.010	0.03	0.05	-	+/-
6223-10	0.31	0.24	0.55	0.012	0.03	0.04	-	+/-
6223-11	0.28	0.25	0.56	0.008	0.02	0.06	-	+/-
6223-12	0.29	0.25	0.53	0.005	0.03	0.05	-	+/-
6223-13	0.29	0.22	0.58	0.005	0.03	0.07	-	+/-
6223-14	0.29	0.26	0.55	0.007	0.03	0.05	-	+/-
6223-15	0.36	0.28	0.53	0.026	0.04	0.07	-	-
6223-16	0.39	0.34	0.52	0.140	0.10	0.07	-	+/-
6223-17	1.11	1.21	0.52	1.086	0.62	0.06	-	+/-
6223-18	1.42	1.90	0.55	3.091	1.03	0.08	-	+/-
6223-19	2.19	2.10	0.56	3.069	1.62	0.08	-	+/-
6223-20	2.16	2.49	0.53	3.957	1.55	0.08	-	+/-
6223-21	1.78	2.45	0.63	2.595	1.28	0.06	-	+/-
6223-22	1.39	2.03	0.58	2.267	1.15	0.07	-	+/-

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)



**HCV Seroconversion Panel Donor No. 62507**  
**Part Number: HCV6223**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO RIBA 3.0 C22	ORTHO RIBA 3.0 NS5	ORTHO RIBA 3.0 SOD	PCR RNA Qualitative Result Lab # 3	PCR RNA Quantitative Result Lab # 4 (Copies/mL)	ORTHO ECI HBsAg	ORTHO ECI ANTI-HCV	ORTHO ECI ANTI-HIV 1/2
	TEST DATE: 9/25/1997	TEST DATE: 9/25/1997	TEST DATE: 9/25/1997	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
6223-01	-	-	-	NOT DETECTED	<400	0.06	0.01	0.14
6223-02	-	-	-	NOT DETECTED	<400	0.06	0.01	0.13
6223-03	-	-	-	NOT DETECTED	<400	0.06	0.01	0.14
6223-04	-	-	-	NOT DETECTED	<400	0.06	0.01	0.13
6223-05	-	-	-	NOT DETECTED	<400	0.07	0.01	0.14
6223-06	-	-	-	NOT DETECTED	<400	0.05	0.01	0.14
6223-07	-	-	-	NOT DETECTED	<400	0.07	0.01	0.13
6223-08	-	-	-	NOT DETECTED	<400	0.06	0.01	0.13
6223-09	-	-	-	NOT DETECTED	<400	0.07	0.01	0.13
6223-10	-	-	-	NOT DETECTED	<400	0.06	0.01	0.14
6223-11	-	-	-	NOT DETECTED	<400	0.06	0.01	0.13
6223-12	-	-	-	NOT DETECTED	<400	0.06	0.01	0.14
6223-13	-	-	-	NOT DETECTED	<400	0.06	0.01	0.14
6223-14	-	-	-	NOT DETECTED	<400	0.05	0.01	0.16
6223-15	-	-	-	NOT DETECTED	<400	0.05	0.07	0.15
6223-16	1	-	-	NOT DETECTED	<400	0.06	0.62	0.15
6223-17	3	-	-	NOT DETECTED	<400	0.06	3.56	0.15
6223-18	4	-	-	NOT DETECTED	<400	0.06	5.20	0.14
6223-19	4	-	-	NOT DETECTED	<400	0.06	6.34	0.15
6223-20	4	-	-	NOT DETECTED	<400	0.05	6.57	0.14
6223-21	4	-	-	NOT DETECTED	<400	0.07	6.37	0.14
6223-22	4	-	-	NOT DETECTED	<400	0.06	5.55	0.15

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHCV6223 Rev03  
 Effective Date: 10/08/2021