



HCV Seroconversion Panel Donor No. 63135
Part Number: HCV6226

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

| PANEL MEMBER | DATE OF DRAW | ALT | ABBOTT MUREX ANTI-HCV 4.0 | ABBOTT MUREX HCV Ag/Ab Combo | ABBOTT Corzyme ANTI-HBc | ABBOTT ANTI-HCV 2.0 Lab # 1 | ABBOTT Architect ANTI-HCV | ABBOTT PRISM ANTI-HCV |
|--------------|--------------|--------------------------|---------------------------|------------------------------|-------------------------|-----------------------------|---------------------------|-----------------------|
| | | Normal Range (10-45 u/L) | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK |
| 6226-01 | 12/17/96 | 16 | 0.14 | 0.473 | 0.11 | 0.336 | 0.07 | 0.102 |
| 6226-02 | 12/20/96 | 19 | 0.15 | 0.375 | 0.11 | 0.179 | 0.06 | 0.112 |
| 6226-03 | 12/26/96 | 9 | 0.16 | 0.527 | 0.10 | 0.164 | 0.07 | 0.102 |
| 6226-04 | 12/28/96 | 11 | 0.16 | 0.429 | 0.12 | 0.145 | 0.06 | 0.089 |
| 6226-05 | 01/09/97 | 37 | 0.16 | 0.314 | 0.10 | 0.189 | 0.07 | 0.097 |
| 6226-06 | 01/11/97 | 19 | 0.15 | 0.399 | 0.12 | 0.038 | 0.07 | 0.101 |
| 6226-07 | 01/16/97 | 19 | 0.15 | 0.456 | 0.10 | 0.162 | 0.15 | 0.188 |
| 6226-08 | 01/18/97 | 25 | 0.16 | 0.402 | 0.09 | 0.202 | 0.23 | 0.342 |
| 6226-09 | 01/23/97 | 37 | 0.76 | 0.412 | 0.12 | 0.126 | 2.27 | 2.633 |
| 6226-10 | 01/25/97 | 40 | 2.32 | 0.456 | 0.11 | 0.155 | 4.52 | 4.867 |
| 6226-11 | 01/30/97 | 60 | 5.13 | 0.622 | 0.11 | 0.176 | 9.96 | 7.948 |
| 6226-12 | 02/02/97 | 52 | 5.29 | 0.939 | 0.11 | 0.195 | 10.82 | 8.769 |

| PANEL MEMBER | ROCHE DIAGNOSTICS Cobas ANTI-HCV | ORTHO ELISA ANTI-HCV 3.0 | ORTHO RIBA 2.0 | ORTHO RIBA 3.0 C100 | ORTHO RIBA 3.0 C33 | ORTHO RIBA 3.0 C22 | ORTHO RIBA 3.0 NS5 | ORTHO RIBA 3.0 SOD |
|--------------|----------------------------------|--------------------------|----------------|-----------------------|-----------------------|-----------------------|-----------------------|-----------------------|
| | TEST DATE: 11/7/1997 | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: 10/29/1997 | TEST DATE: 10/29/1997 | TEST DATE: 10/29/1997 | TEST DATE: 10/29/1997 | TEST DATE: 10/29/1997 |
| 6226-01 | 0.16 | 0.005 | - | - | - | - | - | - |
| 6226-02 | 0.12 | 0.002 | - | - | - | - | - | - |
| 6226-03 | 0.11 | 0.002 | - | - | - | - | - | - |
| 6226-04 | 0.12 | 0.003 | - | - | - | - | - | - |
| 6226-05 | 0.10 | 0.002 | - | - | - | - | - | - |
| 6226-06 | 0.16 | 0.003 | - | - | - | - | - | - |
| 6226-07 | 0.11 | 0.003 | - | - | - | - | - | - |
| 6226-08 | 0.12 | 0.007 | - | - | - | - | - | - |
| 6226-09 | 0.14 | 0.375 | - | - | - | - | - | - |
| 6226-10 | 0.17 | 1.218 | - | - | +/- | - | - | - |
| 6226-11 | 0.24 | 4.016 | - | +/- | 2+ | - | - | - |
| 6226-12 | 0.60 | 4.072 | - | +/- | 2+ | - | - | - |

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com



**HCV Seroconversion Panel Donor No. 63135
Part Number: HCV6226**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

| PANEL MEMBER | CHIRON bDNA (MEq/mL) TEST DATE: 9/23/1997 | ROCHE HCV MONITOR (copies/mL) TEST DATE: UNK | PCR RNA Quantitative Result Lab # 3 TEST DATE: UNK | ZEPTOMETRIX HCV RT PCR TEST DATE: UNK | ORTHO ECi HBsAg TEST DATE: 11/16/2018 | ORTHO ECi ANTI-HCV TEST DATE: 11/16/2018 | ORTHO ECi ANTI-HIV ½ TEST DATE: 11/16/2018 |
|--------------|---|---|---|--|---|--|---|
| 6226-01 | 2.379 | 230000 | 1900000 | 254000 | 0.05 | 0.00 | 0.13 |
| 6226-02 | 0.852 | 65000 | 670000 | 58300 | 0.05 | 0.00 | 0.13 |
| 6226-03 | 3.788 | 160000 | 710000 | 196000 | 0.05 | 0.01 | 0.12 |
| 6226-04 | 1.542 | 140000 | 730000 | 362000 | 0.06 | 0.01 | 0.14 |
| 6226-05 | 0.232 | 2500 | 50000 | 27300 | 0.07 | 0.01 | 0.14 |
| 6226-06 | 2.382 | 170000 | 740000 | 158000 | 0.05 | 0.01 | 0.13 |
| 6226-07 | 2.367 | 250000 | 1100000 | 334000 | 0.05 | 0.01 | 0.12 |
| 6226-08 | 1.915 | 140000 | 1300000 | 174000 | 0.05 | 0.02 | 0.12 |
| 6226-09 | 1.184 | 74000 | 580000 | 107000 | 0.06 | 1.01 | 0.13 |
| 6226-10 | 0.955 | 54000 | 250000 | 169000 | 0.05 | 2.47 | 0.11 |
| 6226-11 | 0.402 | 41000 | 320000 | 113000 | 0.05 | 6.37 | 0.12 |
| 6226-12 | 0.283 | 31000 | 310000 | 53900 | 0.05 | 7.22 | 0.13 |

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com