



**HCV Seroconversion Panel Donor No. 63318**  
**Part Number: HCV6227**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)  TEST DATE: UNK	ABBOTT MUREX ANTI-HCV 4.0  TEST DATE: UNK	ABBOTT MUREX HCV Ag/Ab Combo  TEST DATE: UNK	ABBOTT Corzyme ANTI-HBc  TEST DATE: UNK	ABBOTT ANTI-HCV 2.0 Lab # 1  TEST DATE: UNK	ABBOTT ANTI-HCV 2.0 Lab # 2  TEST DATE: 11/6/1997	ABBOTT Architect ANTI-HCV  TEST DATE: UNK	ABBOTT PRISM ANTI-HCV  TEST DATE: UNK
6227-01	12/17/96	6	0.14	0.419	2.23	0.204	0.17	0.07	0.300
6227-02	01/08/97	14	0.15	0.405	2.22	0.231	0.19	0.06	0.310
6227-03	01/10/97	3	0.14	0.368	2.27	0.183	0.18	0.08	0.300
6227-04	01/28/97	34	0.15	1.784	2.00	0.235	0.20	0.09	0.330
6227-05	02/01/97	23	0.14	4.534	1.94	0.218	0.22	0.07	0.340
6227-06	03/01/97	59	4.40	11.696	1.99	0.855	0.86	6.55	3.690
6227-07	03/03/97	212	5.23	11.892	2.22	1.376	1.45	7.41	3.990

PANEL MEMBER	ROCHE DIAGNOSTICS Cobas ANTI- HCV  TEST DATE: 11/7/1997	ABBOTT AxSYM ANTI-HCV  TEST DATE: 10/21/1997	ORTHO ANTI-HCV 3.0 (SAVe)  TEST DATE: 10/14/1997	ORTHO ELISA ANTI-HCV 3.0  TEST DATE: UNK	ORTHO RIBA 2.0  TEST DATE: UNK	ORTHO RIBA 3.0 C100  TEST DATE: 10/29/1997	ORTHO RIBA 3.0 C33  TEST DATE: 10/29/1997	ORTHO RIBA 3.0 C22  TEST DATE: 10/29/1997
6227-01	0.15	0.63	0.00	0.029	-	-	-	-
6227-02	0.10	0.63	0.00	0.067	-	-	-	-
6227-03	0.12	0.52	0.00	0.028	-	-	-	-
6227-04	0.16	0.56	0.01	0.042	-	-	-	-
6227-05	0.14	0.53	0.00	0.028	-	-	-	-
6227-06	0.99	1.75	2.10	3.686	-	+/-	3+	2+
6227-07	5.56	1.64	2.52	3.730	INDETERMINATE	+/-	3+	4+

PANEL MEMBER	ORTHO RIBA 3.0 NS5  TEST DATE: 10/29/1997	ORTHO RIBA 3.0 SOD  TEST DATE: 10/29/1997	CHIRON bDNA (MEq/mL)  TEST DATE: 9/23/1997	ROCHE HCV MONITOR (copies/mL)  TEST DATE: UNK	PCR RNA Quantitative Result Lab # 3  TEST DATE: 10/25/1997	ORTHO ECi HBsAg  TEST DATE: 11/16/2018	ORTHO ECi ANTI-HCV  TEST DATE: 11/16/2018	ORTHO ECi ANTI-HIV 1/2  TEST DATE: 11/16/2018
6227-01	-	NEGATIVE	0.226	<166	<100	0.06	0.03	0.11
6227-02	-	NEGATIVE	<0.2	<166	<100	0.05	0.03	0.11
6227-03	-	NEGATIVE	<0.2	<166	<100	0.05	0.05	0.12
6227-04	-	NEGATIVE	11.18	390000	2100000	0.06	0.05	0.11
6227-05	-	NEGATIVE	31.05	770000	>5000000	0.06	0.04	0.12
6227-06	1+	NEGATIVE	55.14	820,000	>5000000	0.07	6.17	0.12
6227-07	+/-	NEGATIVE	46.75	650,000	1300000	0.06	7.82	0.13

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)