



**HCV Seroconversion Panel Donor No. 62886**  
**Part Number: HCV6228**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

**HCV Genotype: 1a**

| PANEL MEMBER | DATE OF DRAW | ALT<br>Normal Range<br>(10-45 u/L) | ABBOTT<br>MUREX<br>ANTI-HCV<br>4.0 | ABBOTT<br>MUREX<br>HCV Ag/Ab<br>Combo | ABBOTT<br>Corzyme<br>ANTI-HBc | ABBOTT<br>ANTI-HCV<br>2.0<br>Lab # 1 | ABBOTT<br>ANTI-HCV<br>2.0<br>Lab # 2 | ABBOTT<br>Architect<br>ANTI-HCV | ABBOTT<br>PRISM<br>ANTI-HCV |
|--------------|--------------|------------------------------------|------------------------------------|---------------------------------------|-------------------------------|--------------------------------------|--------------------------------------|---------------------------------|-----------------------------|
|              |              | TEST DATE:<br>UNK                  | TEST DATE:<br>UNK                  | TEST DATE:<br>UNK                     | TEST DATE:<br>UNK             | TEST DATE:<br>UNK                    | TEST DATE:<br>11/6/1997              | TEST DATE:<br>UNK               | TEST DATE:<br>UNK           |
| 6228-01      | 11/20/96     | 41                                 | 0.142                              | 0.584                                 | 1.60                          | 0.132                                | 0.08                                 | 0.030                           | 0.062                       |
| 6228-02      | 11/22/96     | 26                                 | 0.127                              | 0.416                                 | 1.42                          | 0.134                                | 0.11                                 | 0.040                           | 0.081                       |
| 6228-03      | 11/27/96     | 32                                 | 0.134                              | 1.027/0.936                           | 1.31                          | 0.145                                | 0.13                                 | 0.040                           | 0.078                       |
| 6228-04      | 11/29/96     | 36                                 | 0.128                              | 0.580                                 | 1.46                          | 0.120                                | 0.12                                 | 0.030                           | 0.066                       |
| 6228-05      | 12/04/96     | 100                                | 0.130                              | 0.352                                 | 1.24                          | 0.155                                | 0.13                                 | 0.040                           | 0.076                       |
| 6228-06      | 12/06/96     | 248                                | 0.110                              | 0.297                                 | 1.50                          | 0.128                                | 0.11                                 | 0.030                           | 0.080                       |
| 6228-07      | 12/11/96     | 47                                 | 0.131                              | 0.485                                 | 1.80                          | 0.143                                | 0.12                                 | 0.090                           | 0.163                       |
| 6228-08      | 12/14/96     | 196                                | 0.124                              | 0.608                                 | 1.72                          | 0.128                                | 0.14                                 | 0.100                           | 0.137                       |
| 6228-09      | 12/18/96     | 156                                | 0.422                              | 0.556                                 | 1.72                          | 0.181                                | 0.19                                 | 1.370                           | 1.392                       |
| 6228-10      | 12/21/96     | 53                                 | 0.985                              | 1.092                                 | 1.59                          | 0.252                                | 0.21                                 | 4.520                           | 4.582                       |
| 6228-11      | 12/26/96     | 100                                | 1.673                              | 1.666                                 | 1.35                          | 0.290                                | 0.32                                 | 6.620                           | 6.224                       |
| 6228-12      | 12/28/96     | 96                                 | 1.679                              | 1.529                                 | 1.44                          | 0.370                                | 0.31                                 | 7.120                           | 6.665                       |

| PANEL MEMBER | ROCHE<br>DIAGNOSTICS<br>Cobas<br>ANTI-HCV | ABBOTT<br>AxSYM<br>ANTI-HCV | ZEPTOMETRIX<br>HCVRT<br>PCR | ORTHO<br>ANTI-HCV<br>3.0<br>(SAVe) | ORTHO<br>ELISA<br>ANTI-HCV<br>3.0 | ORTHO<br>ECi<br>ANTI-HIV | ORTHO<br>RIBA<br>2.0 | ORTHO<br>RIBA<br>3.0<br>C100 |
|--------------|---|-----------------------------|-----------------------------|------------------------------------|-----------------------------------|--------------------------|----------------------|------------------------------|
|              | TEST DATE:<br>11/7/1997                   | TEST DATE:<br>10/21/1997    | TEST DATE:<br>UNK           | TEST DATE:<br>10/14/1997           | TEST DATE:<br>UNK                 | TEST DATE:<br>UNK        | TEST DATE:<br>UNK    | TEST DATE:<br>UNK            |
| 6228-01      | 0.09                                      | 0.52                        | 150000                      | 0.01                               | 0.000                             | 0.03                     | -                    | -                            |
| 6228-02      | 0.09                                      | 0.47                        | 46300                       | 0.01                               | 0.000                             | 0.03                     | -                    | -                            |
| 6228-03      | 0.1                                       | 0.46                        | 138000                      | 0.01                               | 0.002                             | 0.03                     | -                    | -                            |
| 6228-04      | 0.1                                       | 0.50                        | 49600                       | 0.01                               | 0.000                             | 0.03                     | -                    | -                            |
| 6228-05      | 0.09                                      | 0.47                        | 35400                       | 0.01                               | 0.000                             | 0.03                     | -                    | -                            |
| 6228-06      | 0.1                                       | 0.47                        | 1840                        | 0.01                               | 0.010                             | 0.03                     | -                    | -                            |
| 6228-07      | 0.1                                       | 0.54                        | 61000                       | 0.01                               | 0.007                             | 0.05                     | -                    | -                            |
| 6228-08      | 0.09                                      | 0.51                        | 169000                      | 0.01                               | 0.007                             | 0.08                     | -                    | -                            |
| 6228-09      | 0.06                                      | 0.52                        | 407                         | 0.28                               | 0.642                             | 1.33                     | -                    | -                            |
| 6228-10      | 0.1                                       | 0.63                        | 816                         | 1.10                               | 2.166                             | 4.03                     | -                    | -                            |
| 6228-11      | 0.15                                      | 0.62                        | 38800                       | 1.31                               | 2.842                             | 4.50                     | -                    | -                            |
| 6228-12      | 0.17                                      | 0.62                        | 3160                        | 1.48                               | 3.192                             | 4.84                     | -                    | -                            |

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)



**HCV Seroconversion Panel Donor No. 62886**  
**Part Number: HCV6228**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

| PANEL MEMBER | ORTHO RIBA 3.0 C33 | ORTHO RIBA 3.0 C22 | ORTHO RIBA 3.0 NS5 | ORTHO RIBA 3.0 SOD | ORTHO RIBA 3.0        | CHIRON bDNA (MEq/mL) | ROCHE HCV MONITOR (copies/mL) | PCR RNA Quantitative Result Lab # 3 |
|--------------|--------------------|--------------------|--------------------|--------------------|-----------------------|----------------------|-------------------------------|-------------------------------------|
|              | TEST DATE: UNK     | TEST DATE: UNK     | TEST DATE: UNK     | TEST DATE: UNK     | TEST DATE: 10/29/1997 | TEST DATE: 9/23/1997 | TEST DATE: UNK                | TEST DATE: 10/25/1997               |
| 6228-01      | -                  | -                  | -                  | -                  | -                     | 1.486                | 270000                        | 670000                              |
| 6228-02      | -                  | -                  | -                  | -                  | -                     | 0.986                | 180000                        | 260000                              |
| 6228-03      | -                  | -                  | -                  | -                  | -                     | 2.480                | 320000                        | 1000000                             |
| 6228-04      | -                  | -                  | -                  | -                  | -                     | 0.916                | 400000                        | 445000                              |
| 6228-05      | -                  | -                  | -                  | -                  | -                     | <0.2                 | 99000                         | 240000                              |
| 6228-06      | -                  | -                  | -                  | -                  | -                     | <0.2                 | 5300                          | 15500                               |
| 6228-07      | -                  | -                  | -                  | -                  | -                     | 0.898                | 160000                        | 1000000                             |
| 6228-08      | -                  | -                  | -                  | -                  | -                     | 1.625                | 140000                        | 1300000                             |
| 6228-09      | -                  | -                  | -                  | -                  | -                     | <0.2                 | 340                           | 500                                 |
| 6228-10      | 1+                 | -                  | -                  | -                  | INDETERMINATE         | <0.2                 | 1600                          | 1000                                |
| 6228-11      | 2+                 | -                  | -                  | -                  | INDETERMINATE         | 0.396                | 78000                         | 440000                              |
| 6228-12      | 2+                 | -                  | -                  | -                  | INDETERMINATE         | <0.2                 | 3000                          | 7500                                |

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)