



HCV Seroconversion Panel Donor No. 64150
Part Number: HCV9045

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ABBOTT MUREX ANTI-HCV 4 0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1	ABBOTT ANTI-HCV Lab # 2	ABBOTT ANTI-HCV 3.0	ORTHO ANTI-HCV 3.0	ABBOTT PRISM HCV
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 5/20/1998	TEST DATE: 6/15/1998	TEST DATE: 7/8/1998	TEST DATE: UNK	TEST DATE: 8/5/1998
9045-01	05/09/97	27	0.084	4.833	0.043	0.112	0.07	0.008	0.06
9045-02	05/11/97	22	0.087	5.038	0.043	0.109	0.09	0.005	0.06
9045-03	05/16/97	30	0.095	5.587	0.043	0.128	0.11	0.007	0.07
9045-04	05/18/97	14	0.090	5.952	0.043	0.132	0.07	0.005	0.07
9045-05	06/04/97	27	0.107	6.119	0.065	0.155	0.11	0.007	0.06
9045-06	06/10/97	216	0.152	3.130	0.108	0.144	0.22	0.005	0.12
9045-07	06/15/97	304	1.213	5.205	0.240	0.399	0.34	0.654	2.03
9045-08	06/19/97	308	1.828	3.802	0.455	0.838	0.92	1.391	3.24

PANEL MEMBER	ABBOTT IMx HCV 3.0 Reagent Pack	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV	ROCHE/BOEHRINGER MANNHEIM Cobas Core ANTI-HCV EIA	ORTHO ECI ANTI-HCV	ORTHO Enhanced SAVE HCV 3.0 Lab # 4	ORTHO Enhanced SAVE HCV 3.0 Lab # 7
	TEST DATE: 8/4/1998	TEST DATE: 7/6/1998	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 7/7/1998	TEST DATE: UNK	TEST DATE: 6/29/1998	TEST DATE: 6/23/1998
9045-01	0.12	0.14	0.040	0.060	0.1	0.03	0.00	0.016
9045-02	0.12	0.13	0.030	0.060	0.1	0.03	0.00	0.022
9045-03	0.14	0.11	0.040	0.060	0.1	0.03	0.00	0.006
9045-04	0.15	NOT TESTED	0.030	0.060	0.1	0.03	0.01	0.003
9045-05	0.13	0.17	0.040	0.070	0.1	0.04	0.00	0.000
9045-06	0.23	0.23	0.120	0.120	0.1	0.06	0.04	0.025
9045-07	1.28	2.39	3.410	2.240	0.5	2.68	2.61	1.599
9045-08	2.36	4.11	5.300	3.680	1.2	3.84	3.51	2.430

PANEL MEMBER	SANOFI Monolisa ANTI- HCV PLUS	SANOFI Monolisa ANTI- HCV PLUS Lab # 7	ORTHO Chiron Riba HCV 2.0 5-1-1(NS4)	ORTHO Chiron Riba HCV 2.0 C100(NS4)	ORTHO Chiron Riba HCV 2.0 C33(NS3)	ORTHO Chiron Riba HCV 2.0 C22(Core)	ORTHO Chiron Riba HCV 2.0 SOD)	ORTHO Chiron Riba HCV 3.0 c100(p)
	TEST DATE: 6/30/1998	TEST DATE: 6/23/1998	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
9045-01	0.07	0.028	-	-	-	-	-	-
9045-02	0.06	0.037	-	-	-	-	-	-
9045-03	0.05	0.049	-	-	-	-	-	-
9045-04	0.05	0.042	-	-	-	-	-	-
9045-05	0.08	0.034	-	-	-	-	-	-
9045-06	0.25	0.069	-	-	-	-	-	-
9045-07	2.12	0.729	-	-	+/-	+/-	-	-
9045-08	2.89	2.399	+/-	+/-	1+	+/-	-	+/-

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com



HCV Seroconversion Panel Donor No. 64150
Part Number: HCV9045

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO Chiron Riba HCV 3.0 c33c TEST DATE: UNK	ORTHO Chiron Riba HCV 3.0 c22(p) TEST DATE: UNK	ORTHO Chiron Riba HCV 3.0 NS5 TEST DATE: UNK	ORTHO Chiron Riba HCV 3.0 Interpretation TEST DATE: UNK	CHIRON HCV RNA bDNA 2.0 (MEq/mL*) TEST DATE: 7/16/1998	ORTHO ECi HBsAg TEST DATE: 11/16/2018	ORTHO ECi ANTI-HCV TEST DATE: 11/16/2018	ORTHO ECi ANTI-HIV ½ TEST DATE: 11/16/2018
9045-01	-	-	-	-	17.73	0.05	0.01	0.13
9045-02	-	-	-	-	16.42	0.05	0.01	0.13
9045-03	-	-	-	-	36.19	0.05	0.01	0.12
9045-04	-	-	-	-	38.07	0.05	0.01	0.13
9045-05	-	-	-	-	44.86	0.06	0.01	0.14
9045-06	-	-	-	-	17.78	0.06	0.02	0.13
9045-07	1+	-	-	-	18.36	0.06	1.41	0.13
9045-08	2+	-	-	INDETERMINATE	2.322	0.05	1.82	0.12

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com