



HCV Seroconversion Panel Donor No. 64273
Part Number: HCV9047

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L) TEST DATE: UNK	ABBOTT MUREX ANTI-HCV 4 0 TEST DATE: UNK	ABBOTT MUREX HCV Ag/Ab Combo TEST DATE: UNK	ABBOTT ANTI-HCV Lab # 1 TEST DATE: 5/20/1998	ABBOTT ANTI-HCV Lab # 2 TEST DATE: 6/15/1998	ABBOTT ANTI-HCV 3.0 TEST DATE: 7/8/1998	ORTHO ANTI-HCV 3.0 TEST DATE: UNK
9047-01	06/04/97	11	0.142	8.840	0.108	0.189	0.14	0.008
9047-02	06/06/97	5	0.148	10.014	0.108	0.235	0.16	0.012
9047-03	06/14/97	7	0.136	7.945	0.108	0.200	0.21	0.008
9047-04	06/16/97	3	0.124	8.655	0.152	0.182	0.16	0.010
9047-05	06/23/97	26	0.103	6.478	0.108	0.194	0.18	0.008
9047-06	06/25/97	26	0.136	5.102	0.108	0.187	0.18	0.012
9047-07	07/02/97	324	0.944	6.038	0.195	0.305	0.36	1.286
9047-08	07/04/97	272	2.849	12.167	0.998	0.977	1.91	4.139
9047-09	07/09/97	252	5.472	6.007	2.230	2.772	4.25	4.139
9047-10	07/11/97	276	5.421	6.389	2.857	3.016	1.00	4.139

PANEL MEMBER	ABBOTT PRISM HCV TEST DATE: 8/5/1998	ABBOTT IMx HCV 3.0 Reagent Pack TEST DATE: 8/4/1998	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack TEST DATE: 7/6/1998	ABBOTT Architect ANTI-HCV TEST DATE: UNK	ABBOTT PRISM ANTI-HCV TEST DATE: UNK	ROCHE/ BOEHRINGER MANNHEIM Cobas Core ANTI-HCV EIA TEST DATE: 7/7/1998	ORTHO Enhanced SAVe HCV 3.0 Lab # 4 TEST DATE: 7/15/1998	ORTHO Enhanced SAVe HCV 3.0 Lab # 7 TEST DATE: 6/25/1998
9047-01	0.09	0.13	0.16	0.060	0.080	0.4	0.01	0.102
9047-02	0.10	0.20	0.13	0.070	0.070	0.4	0.01	0.122
9047-03	0.09	0.15	0.17	0.060	0.070	0.4	0.01	0.086
9047-04	0.09	0.20	0.16	0.060	0.070	0.3	0.01	0.052
9047-05	0.11	0.20	0.18	0.060	0.080	0.4	0.01	0.102
9047-06	0.13	0.24	0.19	0.070	0.080	0.4	0.01	0.090
9047-07	2.47	3.29	6.40	4.840	2.850	0.6	1.78	1.513
9047-08	3.88	16.25	26.79	9.600	4.960	2.7	3.70	3.924
9047-09	4.79	26.75	43.67	10.170	5.840	9.8	MAX. ABS.	7.355
9047-10	5.01	28.21	48.08	9.940	6.640	9.2	MAX. ABS.	6.770

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com



**HCV Seroconversion Panel Donor No. 64273
Part Number: HCV9047**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	SANOFI Monolisa ANTI- HCV PLUS	SANOFI Monolisa ANTI- HCV PLUS Lab # 7	ORTHO Chiron Riba HCV 2.0 5-1-1(NS4)	ORTHO Chiron Riba HCV 2.0 C100(NS4)	ORTHO Chiron Riba HCV 2.0 C33(NS3)	ORTHO Chiron Riba HCV 2.0 C22(Core)	ORTHO Chiron Riba HCV 2.0 SOD	ORTHO Chiron Riba HCV 3.0 c100(p)
	TEST DATE: 6/30/1998	TEST DATE: 6/25/1998	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
9047-01	0.07	0.091	-	+/-	+/-	-	-	-
9047-02	0.07	0.123	-	+/-	+/-	-	-	-
9047-03	0.07	0.103	-	+/-	+/-	-	-	-
9047-04	0.11	0.067	-	+/-	+/-	-	-	-
9047-05	0.07	0.119	-	+/-	+/-	-	-	-
9047-06	0.07	0.155	-	+/-	+/-	-	-	-
9047-07	2.50	0.725	-	+/-	+/-	-	-	-
9047-08	5.89	4.119	-	+/-	2+	-	-	-
9047-09	6.51	5.888	+/-	+/-	4+	-	-	+/-
9047-10	7.10	6.448	+/-	+/-	3+	-	-	2+

PANEL MEMBER	ORTHO Chiron Riba HCV 3.0 c33c	ORTHO Chiron Riba HCV 3.0 c22(p)	ORTHO Chiron Riba HCV 3.0 NS5	ORTHO Chiron Riba HCV 3.0 Interpretation	CHIRON HCV RNA bDNA 2.0 (MEq/mL)	ORTHO ECi HBsAg	ORTHO ECi ANTI-HCV	ORTHO ECi ANTI-HIV ½
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 7/16/1998	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
9047-01	-	-	-	NEGATIVE	65.95	0.05	0.02	0.13
9047-02	-	-	-	NEGATIVE	81.76	0.05	0.01	0.12
9047-03	-	-	-	NEGATIVE	54.85	0.06	0.02	0.12
9047-04	-	-	-	NEGATIVE	64.03	0.05	0.02	0.11
9047-05	-	-	-	NEGATIVE	55.45	0.06	0.03	0.11
9047-06	-	-	-	NEGATIVE	42.97	0.06	0.02	0.11
9047-07	2+	-	-	INDETERMINATE	9.132	0.05	2.27	0.13
9047-08	4+	-	-	INDETERMINATE	26.58	0.06	6.32	0.12
9047-09	4+	-	-	INDETERMINATE	22.77	0.06	11.0	0.12
9047-10	4+	+/-	+/-	POSITIVE	NOT DETECTED	0.05	12.0	0.11

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com