



HCV Seroconversion Panel Donor No. 64766
Part Number: HCV9049

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L) TEST DATE: UNK	ABBOTT MUREX ANTI-HCV 4 0 TEST DATE: UNK	ABBOTT MUREX HCV Ag/Ab Combo TEST DATE: UNK	ABBOTT ANTI-HCV Lab # 1 TEST DATE: 5/20/1998	ABBOTT ANTI-HCV Lab # 2 TEST DATE: 6/15/1998	ABBOTT ANTI-HCV 3.0 TEST DATE: 7/8/1998	ORTHO Chiron Riba HCV 3.0 TEST DATE: UNK	ABBOTT PRISM HCV TEST DATE: 8/5/1998
9049-01	09/10/97	22	0.330	0.338	0.660	0.660	1.37	0.111	0.25
9049-02	09/12/97	16	0.911	0.345	1.927	2.060	0.84	0.303	0.92
9049-03	09/18/97	18	0.596	0.317	1.163	1.210	0.50	0.296	0.64
9049-04	09/22/97	12	0.407	0.317	0.781	0.793	4.17	0.124	0.40
9049-05	09/26/97	11	0.303	0.328	0.556	0.600	1.63	0.103	0.28

PANEL MEMBER	ABBOTT IMx HCV 3.0 Reagent Pack TEST DATE: 8/4/1998	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack TEST DATE: 7/6/1998	ABBOTT Architect ANTI-HCV TEST DATE: UNK	ABBOTT PRISM ANTI-HCV TEST DATE: UNK	ROCHE/ BOEHRINGER MANNHEIM Cobas Core ANTI-HCV EIA TEST DATE: 7/7/1998	ORTHO Enhanced SAVE HCV 3.0 Lab # 2 TEST DATE: UNK	ORTHO Enhanced SAVE HCV 3.0 Lab # 7 TEST DATE: 7/2/1998	SANOFI Monolisa ANTI- HCV PLUS Lab # 4 TEST DATE: 6/30/1998
9049-01	0.28	0.35	0.390	0.388	0.3	0.140	0.412	1.09
9049-02	0.46	0.77	1.430	1.522	0.4	0.583	2.324	3.45
9049-03	0.32	0.63	0.870	0.881	0.3	0.489	0.882	2.21
9049-04	0.24	0.43	0.550	0.576	0.2	0.340	0.463	1.36
9049-05	0.27	0.27	0.380	0.433	0.2	0.160	0.370	1.06

PANEL MEMBER	SANOFI Monolisa ANTI- HCV PLUS Lab # 7 TEST DATE: 7/2/1998	ORTHO Chiron Riba HCV 2.0 5-1-1(NS4) TEST DATE: UNK	ORTHO Chiron Riba HCV 2.0 C100(NS4) TEST DATE: UNK	ORTHO Chiron Riba HCV 2.0 C33(NS3) TEST DATE: UNK	ORTHO Chiron Riba HCV 2.0 C22(Core) TEST DATE: UNK	ORTHO Chiron Riba HCV 2.0 SOD TEST DATE: UNK	ORTHO Chiron RIBA HCV 2.0 Interpretation TEST DATE: UNK	ORTHO Chiron Riba HCV 3.0 c100(p) TEST DATE: UNK
9049-01	0.507	-	-	1+	-	-	INDETERMINATE	-
9049-02	2.258	-	-	1+	-	-	INDETERMINATE	-
9049-03	1.268	-	-	1+	-	-	INDETERMINATE	-
9049-04	0.678	-	-	1+	-	-	INDETERMINATE	-
9049-05	0.241	-	-	1+	-	-	INDETERMINATE	-

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com



HCV Seroconversion Panel Donor No. 64766
Part Number: HCV9049

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO Chiron Riba HCV 3.0 c33c TEST DATE: UNK	ORTHO Chiron Riba HCV 3.0 c22(p) TEST DATE: UNK	ORTHO Chiron Riba HCV 3.0 NS5 TEST DATE: UNK	ORTHO Chiron Riba HCV 3.0 Interpretation TEST DATE: 7/22/1998	CHIRON HCV RNA bDNA 2.0 (MEq/mL*) TEST DATE: 7/16/1998	ORTHO ECi HBsAg TEST DATE: 11/16/2018	ORTHO ECi ANTI-HCV TEST DATE: 11/16/2018	ORTHO ECi ANTI-HIV ½ TEST DATE: 11/16/2018
9049-01	2+	-	+/-	INDETERMINATE	<0.2	0.05	0.39	0.12
9049-02	3+	-	+/-	INDETERMINATE	<0.2	0.04	1.58	0.10
9049-03	3+	-	+/-	INDETERMINATE	<0.2	0.05	0.73	0.11
9049-04	2+	-	+/-	INDETERMINATE	<0.2	0.05	0.39	0.12
9049-05	1+	-	+/-	INDETERMINATE	<0.2	0.05	0.23	0.11

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com