



**HIV Seroconversion Panel Donor No. 73695**  
**Part Number: HIV12007**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ORTHO VITROS ECi ANTI-HIV 1/2 TEST DATE: 8/6/2015	GENE-PROBE PROCLEIX HIV-1/HCV ASSAY TEST DATE: UNK	ROCHE COBAS AMPLICOR HIV-1 MONITOR TEST DATE: UNK	VIROMED PCR TEST DATE: 7/10/2000	ABBOTT HIV-1 p24 Ag TEST DATE: 6/30/2000	ABBOTT IMx HIV ½ III TEST DATE: 9/17/2002	ABBOTT AxSYM HIV Ab ½ EIA TEST DATE: 9/10/2002	ABBOTT MUREX HIV Ab ½ + O (s/Co) TEST DATE: 9/16/2002
12007-01	06/27/99	0.15	NEGATIVE	NEGATIVE	0	0.310	0.49	0.41	0.25
12007-02	06/29/99	0.14	NEGATIVE	NEGATIVE	0	0.293	0.43	0.41	0.22
12007-03	08/20/99	0.12	NEGATIVE	NEGATIVE	0	0.293	0.70	0.59	0.21
12007-04	10/22/99	0.3	POSITIVE	316000	75815	0.897	0.54	1.95	0.26
12007-05	10/24/99	2.97	POSITIVE	118000	63710	0.862	1.10	2.28	1.18
12007-06	10/29/99	60.1	POSITIVE	54600	14749	0.569	10.51	17.91	13.29
12007-07	10/31/99	55.3	POSITIVE	33500	29564	0.414	9.41	14.49	13.08
12007-08	11/05/99	38.7	POSITIVE	300000	18875	0.328	7.54	9.44	13.09
12007-09	11/07/99	32.8	POSITIVE	26400	23054	0.241	7.85	9.24	13.29

PANEL MEMBER	ORTHO VITROS ANTI- HIV ½ TEST DATE: 8/13/2002	TRINITY BIOTECH UNIGOLD RECOMBIGN HIV-1 TEST DATE: UNK	GENETICS SYSTEMS rLAV EIA TEST DATE: 7/6/2000	MEDMIRA ANTI-HIV-1 RAPID TEST TEST DATE: UNK	ORAQUICK ADVANCE HIV ½ Ab RAPID TEST TEST DATE: UNK	BIO-RAD HIV-1 WESTERN BLOT TEST DATE: UNK
12007-01	0.10	NEGATIVE	0.123	NEGATIVE	NEGATIVE	NOT TESTED
12007-02	0.09	NEGATIVE	0.130	NEGATIVE	NEGATIVE	NOT TESTED
12007-03	0.09	NEGATIVE	0.120	NEGATIVE	NEGATIVE	NEGATIVE
12007-04	0.17	NEGATIVE	0.113	NEGATIVE	NEGATIVE	NEGATIVE
12007-05	1.13	NEGATIVE	0.130	NEGATIVE	NEGATIVE	NEGATIVE
12007-06	30.30	POSITIVE	0.933	POSITIVE	NEGATIVE	P55,P40,P24,P18+/-
12007-07	27.40	POSITIVE	1.303	POSITIVE	NEGATIVE	GP160+/-,P55,P40,P24,P18+/-
12007-08	20.90	POSITIVE	1.655	POSITIVE	POSITIVE	GP160+/-,P65,P55,GP41+/-,P40,P31+/-,P24,P18
12007-09	21.20	POSITIVE	1.796	POSITIVE	POSITIVE	GP160+/-,P65,P55,GP41+/-,P40,P31+/-,P24,P18

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)



**HIV Seroconversion Panel Donor No. 73695**  
**Part Number: HIV12007**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT PRISM HIV Ag/Ab COMBO TEST DATE: UNK	ABBOTT MUREX MICROPLATE HIV Ag/Ab COMBO TEST DATE: UNK	ABBOTT ARCHITECT HIV Ag/Ab COMBO TEST DATE: UNK	BIO-RAD MULTISPOT HIV 1/2 RAPID TEST TEST DATE: UNK	BIO-RAD MULTISPOT HIV 1 RAPID TEST TEST DATE: UNK	BIO-RAD MULTISPOT HIV 2 RAPID TEST TEST DATE: UNK
12007-01	0.07	0.276	0.09	NEGATIVE	NEGATIVE	NEGATIVE
12007-02	0.09	0.276	0.13	NEGATIVE	NEGATIVE	NEGATIVE
12007-03	0.06	0.256	0.12	NEGATIVE	NEGATIVE	NEGATIVE
12007-04	2.33	1.281	3.94	NEGATIVE	NEGATIVE	NEGATIVE
12007-05	2.10	1.882	3.35	NEGATIVE	NEGATIVE	NEGATIVE
12007-06	4.71	14.773	14.72	POSITIVE	NEGATIVE	NEGATIVE
12007-07	3.53	12.808	13.50	POSITIVE	NEGATIVE	NEGATIVE
12007-08	2.23	14.419	7.79	POSITIVE	POSITIVE	NEGATIVE
12007-09	2.11	12.813	5.63	POSITIVE	POSITIVE	NEGATIVE

PANEL MEMBER	ABBOTT HIV Ab ½ EIA rDNA (OD/Co) TEST DATE: UNK	ABBOTT PRISM ANTI-HIV ½ +O TEST DATE: 7/15/2002	BIOMERIEUX VIRONOSTIKA ANTI-HIV-1 TEST DATE: UNK	BIO-RAD ANTI-HIV-1 rLAV TEST DATE: UNK	BIOTEST ANTI-HIV TETRA ELISA TEST DATE: 7/1/2002	DADE BEHRING ENZYGNOST ANTI-HIV ½ PLUS TEST DATE: 7/1/2002	ORTHO ANTI-HIV ½ ENHANCED ELISA TEST DATE: 6/24/2002
12007-01	0.095	0.28	0.485	0.069	0.12	0.03	0.00
12007-02	0.129	0.30	0.453	0.055	0.14	0.04	0.00
12007-03	0.121	0.27	0.515	0.044	0.15	0.05	0.00
12007-04	0.129	0.33	0.390	0.047	0.21	0.04	0.03
12007-05	1.147	0.60	0.347	0.051	0.89	0.39	1.23
12007-06	17.241	17.69	0.402	0.724	9.17	OVER	24.40
12007-07	17.241	17.87	0.669	1.204	9.32	OVER	24.14
12007-08	8.379	13.49	0.653	1.764	6.95	7.01	20.00
12007-09	6.957	18.07	0.656	2.309	6.30	7.18	18.10

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)