



HIV Seroconversion Panel Donor No. 73698
Part Number: HIV12008

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ORTHO VITROS Eci ANTI-HIV 1/2 TEST DATE: 8/6/2015	GENE-PROBE PROCLEIX HIV-1/HCV ASSAY TEST DATE: UNK	ROCHE COBAS AMPLICOR HIV-1 MONITOR TEST DATE: UNK	COULTER HIV-1 p24 Ag TEST DATE: 7/5/2000	ABBOTT HIV-1 p24 Ag TEST DATE: 6/29/2000	ABBOTT IMx HIV 1/2 III PLUS TEST DATE: 9/02/2002	ABBOTT AxSYM HIV Ab 1/2 EIA TEST DATE: 6/28/2002
12008-01	09/25/99	0.12	NEGATIVE	NEGATIVE	0.213	0.276	0.27	0.37
12008-02	09/27/99	0.13	NEGATIVE	NEGATIVE	0.160	0.328	0.25	0.36
12008-03	10/02/99	0.13	NEGATIVE	NEGATIVE	0.191	0.241	0.30	0.36
12008-04	10/04/99	0.14	NEGATIVE	NEGATIVE	0.181	0.293	0.30	0.37
12008-05	10/09/99	0.14	NEGATIVE	NEGATIVE	0.191	0.259	0.28	0.36
12008-06	10/11/99	0.14	NEGATIVE	NEGATIVE	0.181	0.293	0.28	0.39
12008-07	10/16/99	0.13	POSITIVE	2000	0.245	0.259	0.26	0.36
12008-08	10/18/99	0.15	POSITIVE	44700	0.511	0.517	0.28	0.39
12008-09	10/23/99	0.16	POSITIVE	>750000	28.723	14.293	0.33	0.38
12008-10	10/28/99	7.59	POSITIVE	>750000	34.043	34.483	1.47	2.39
12008-11	10/30/99	24	POSITIVE	>750000	34.043	9.655	8.42	12.66
12008-12	11/04/99	20.1	POSITIVE	143000	2.053	0.810	8.55	11.65
12008-13	11/06/99	17.9	POSITIVE	NOT TESTED	0.638	0.483	7.11	12.32

PANEL MEMBER	ABBOTT MUREX HIV Ab 1/2 + O TEST DATE: 10/16/2002	ABBOTT HIV Ab 1/2 EIA rDNA TEST DATE: UNK	ABBOTT PRISM ANTI-HIV 1/2 + O TEST DATE: 7/15/2002	ZEPTOMETRIX HIV p24 Ag (pg/mL) TEST DATE: UNK	BIO-RAD ANTI-HIV-1 rLAV TEST DATE: 1/7/2002	BIOTEST ANTI-HIV TETREA ELISA TEST DATE: 7/3/2002	DADE BEHRING ENZYGNOST ANTI-HIV 1/2 PLUS TEST DATE: 7/1/2002	ORTHO ANTI-HIV 1/2 ENHANCED ELISA TEST DATE: 6/24/2002
12008-01	0.27	0.197	0.26	<3.0	0.102	0.14	0.04	0.00
12008-02	0.62	0.181	0.32	<3.0	0.084	0.11	0.04	0.00
12008-03	0.57	0.165	0.26	<3.0	0.084	0.22	0.04	0.01
12008-04	0.28	0.173	0.30	<3.0	0.080	0.17	0.04	0.03
12008-05	0.27	0.213	0.29	<3.0	0.084	0.12	0.03	0.02
12008-06	0.24	0.173	0.32	<3.0	0.065	0.14	0.03	0.01
12008-07	0.24	0.181	0.30	<3.0	0.065	0.12	0.04	0.00
12008-08	0.24	0.157	0.33	<3.0	0.076	0.34	0.04	0.00
12008-09	0.35	0.173	0.33	>125	0.062	0.09	0.04	0.00
12008-10	1.21	1.795	1.77	>125	0.087	1.59	1.09	2.67
12008-11	10.49	9.528	17.93	122.3	0.422	4.51	8.50	10.75
12008-12	13.00	6.063	26.58	<3.0	2.025	3.56	7.75	12.09
12008-13	13.54	4.646	23.25	<3.0	1.713	5.30	7.45	11.16

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com



HIV Seroconversion Panel Donor No. 73698
Part Number: HIV12008

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	MEDMIRA ANTI- HIV-1 RAPID TEST TEST DATE: UNK	TRINITY BIOTECH UNIGOLD RECOMBIGNEN HIV-1 TEST DATE: UNK	BIOMERIEUX VIRONOSTIKA ANTI-HIV-1 TEST DATE: UNK	BOEHRINGER MANNHEIM ENZYMUATEST ANTI-HIV 1/2 PLUS SUBTYPE O TEST DATE: 7/1/2002	ORAQUICK ADVANCE HIV 1/2 RAPID Ab TEST DATE: UNK	BIO-RAD HIV-1 WESTERN BLOT TEST DATE: UNK
12008-01	NEGATIVE	NEGATIVE	0.427	0.04	NEGATIVE	NOT TESTED
12008-02	NEGATIVE	NEGATIVE	0.465	0.04	NEGATIVE	NOT TESTED
12008-03	NEGATIVE	NEGATIVE	0.344	0.04	NEGATIVE	NOT TESTED
12008-04	NEGATIVE	NEGATIVE	0.401	0.04	NEGATIVE	NOT TESTED
12008-05	NEGATIVE	NEGATIVE	0.439	0.03	NEGATIVE	NOT TESTED
12008-06	NEGATIVE	NEGATIVE	0.479	0.03	NEGATIVE	NOT TESTED
12008-07	NEGATIVE	NEGATIVE	0.382	0.04	NEGATIVE	NOT TESTED
12008-08	NEGATIVE	NEGATIVE	0.342	0.04	NEGATIVE	NEGATIVE
12008-09	NEGATIVE	NEGATIVE	0.341	0.04	NEGATIVE	NEGATIVE
12008-10	NEGATIVE	NEGATIVE	0.385	1.09	NEGATIVE	NEGATIVE
12008-11	POSITIVE	POSITIVE	0.371	8.50	NEGATIVE	GP160.P55+/-,P24
12008-12	POSITIVE	POSITIVE	0.633	7.75	POSITIVE	INDETERMINATE
12008-13	POSITIVE	POSITIVE	0.713	7.45	POSITIVE	INDETERMINATE

PANEL MEMBER	BIO-RAD MULTISPOT HIV 1 RAPID TEST TEST DATE: UNK	BIO-RAD MULTISPOT HIV 2 RAPID TEST TEST DATE: UNK	ABBOTT PRISM HIV Ag/Ab COMBO TEST DATE: UNK	ABBOTT MUREX MICROPLATE HIV Ag/Ab COMBO TEST DATE: UNK	ABBOTT ARCHITECT HIV Ag/Ab COMBO TEST DATE: UNK
12008-01	NEGATIVE	NEGATIVE	0.10	0.275	0.08
12008-02	NEGATIVE	NEGATIVE	0.20	0.377	0.14
12008-03	NEGATIVE	NEGATIVE	0.09	0.338	0.09
12008-04	NEGATIVE	NEGATIVE	0.43	0.270	0.11
12008-05	NEGATIVE	NEGATIVE	0.14	0.353	0.10
12008-06	NEGATIVE	NEGATIVE	0.10	0.299	0.10
12008-07	NEGATIVE	NEGATIVE	0.10	0.279	0.13
12008-08	NEGATIVE	NEGATIVE	0.37	0.598	0.53
12008-09	NEGATIVE	NEGATIVE	29.45	14.417	50.26
12008-10	NEGATIVE	NEGATIVE	104.44	17.525	167.70
12008-11	NEGATIVE	NEGATIVE	39.76	17.534	53.66
12008-12	POSITIVE	NEGATIVE	4.23	17.127	28.35
12008-13	POSITIVE	NEGATIVE	3.29	16.990	30.20

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com