



**HIV Seroconversion Panel Donor No. 61694-74**  
**Part Number: HIV6244**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ROCHE COBAS Amplicor HIV-1 Monitor	ABBOTT HIV-1 p24 Ag NT %	ABBOTT HIV-1 p24 Ag	COULTER HIV-1 p24 Ag	COULTER HIV-1 p24 Ag NT%	ABBOTT AxSYM HIV AB ½ EIA	ABBOTT HIV AB ½ EIA rDNA	ABBOTT HIV AB ½ EIA rDNA 3 <sup>rd</sup> Generation
		TEST DATE: UNK	TEST DATE: 1/31/1997	TEST DATE: 1/31/1997	TEST DATE: 1/10/1997	TEST DATE: 1/20/1997	TEST DATE: 2/13/1997	TEST DATE: 12/27/1996	TEST DATE: 2/27/1997
6244-01	05/28/96	<400	NEGATIVE	0.56	0.106	NEGATIVE	0.33	0.558	0.39
6244-02	06/02/96	<400	NEGATIVE	0.53	0.076	NEGATIVE	0.34	0.543	0.43
6244-03	06/04/96	<400	NEGATIVE	0.51	0.091	NEGATIVE	0.33	0.579	0.42
6244-04	06/09/96	<400	NEGATIVE	0.47	0.076	NEGATIVE	0.30	0.574	0.36
6244-05	06/10/96	<400	NEGATIVE	0.65	0.091	NEGATIVE	0.34	0.538	0.35
6244-06	06/11/96	<400	NEGATIVE	0.44	0.121	NEGATIVE	0.34	0.538	0.36
6244-07	06/13/96	<400	NEGATIVE	0.44	0.061	NEGATIVE	0.36	0.518	0.41
6244-08	06/16/96	<400	NEGATIVE	0.42	0.121	NEGATIVE	0.35	0.518	0.36
6244-09	06/18/96	<400	NEGATIVE	0.44	0.076	NEGATIVE	0.32	0.523	0.46
6244-10	06/19/96	<400	NEGATIVE	0.44	0.076	NEGATIVE	0.36	0.574	0.37
6244-11	06/22/96	4550	NEGATIVE	0.46	0.091	NEGATIVE	0.34	0.548	0.38
6244-12	06/23/96	21000	NEGATIVE	0.51	0.136	NEGATIVE	0.33	0.533	0.37
6244-13	06/25/96	274000	92.0	1.33	1.515	98.6	0.32	0.543	0.41
6244-14	06/30/96	>750000	98.3	18.33	27.712	95.0	0.79	2.274	2.17
6244-15	07/02/96	>750000	98.3	13.23	20.364	94.5	3.90	10.152	MAX.ABS.

PANEL MEMBER	ABBOTT PRISM HIV Ag/Ab COMBO 4 <sup>th</sup> Generation	ABBOTT MUREX HIV-1 Ab/Ag COMBO 4 <sup>th</sup> Generation	ABBOTT HIV Ag/Ab COMBO ARCHITECT 4 <sup>th</sup> Generation	ABBOTT MUREX HIV Ab ½ + O 3 <sup>rd</sup> Generation	ABBOTT PRISM ANTI-HIV ½	BIOMERIEUX ORGANON TEKNIKA ANTI-HIV-1	BIOTEST ANTI-HIV ½	DADE BEHRING ENZYGNOST ANTI-HIV ½ PLUS
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 2/19/1997	TEST DATE: UNK	TEST DATE: 2/27/1997	TEST DATE: 2/20/1997
6244-01	0.08	0.245	0.14	0.334	0.30	0.346	0.01	0.09
6244-02	0.08	0.245	0.13	0.338	0.29	0.348	0.02	0.09
6244-03	0.07	0.250	0.18	0.384	0.26	0.323	0.02	0.09
6244-04	0.08	0.250	0.19	0.407	0.21	0.362	0.00	0.08
6244-05	0.09	0.255	0.11	0.344	0.31	0.391	0.01	0.09
6244-06	0.07	0.245	0.13	0.361	0.27	0.389	0.00	0.10
6244-07	0.09	0.235	0.12	0.308	0.21	0.359	0.01	0.25
6244-08	0.07	0.235	0.14	0.315	0.22	0.368	0.01	0.10
6244-09	0.11	0.240	0.08	0.380	0.25	0.343	0.02	0.09
6244-10	0.08	0.235	0.14	0.364	0.25	0.330	0.02	0.10
6244-11	0.10	0.270	0.19	0.390	0.22	0.339	0.05	0.10
6244-12	0.19	0.355	0.25	0.410	0.21	0.314	0.10	0.09
6244-13	1.64	1.725	1.223	0.367	0.26	0.343	0.43	0.10
6244-14	38.51	18.520	30.997	4.774	4.71	0.357	2.24	1.17
6244-15	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED	27.73	0.367	3.77	6.20

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**  
 ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)



**HIV Seroconversion Panel Donor No. 61694-74  
Part Number: HIV6244**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	GENETICS SYSTEMS HIV ½ EIA	CAMBRIDGE BIOTECH HIV-1 WESTERN BLOT	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 <sup>th</sup> Generation	BIO-RAD BioPlex 2200 HIV-1 Ag 5 <sup>th</sup> Generation	BIO-RAD BioPlex 2200 HIV-1 Ab 5 <sup>th</sup> Generation	ORTHO VITROS ECi ANTI-HIV ½
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 8/30/2018
6244-01	0.081	NEGATIVE	0.11	0.11	0.03	0.10
6244-02	0.103	NEGATIVE	0.08	0.08	0.06	0.09
6244-03	0.074	NEGATIVE	0.08	0.08	0.05	0.10
6244-04	0.051	NEGATIVE	0.09	0.09	0.02	0.11
6244-05	0.099	NEGATIVE	0.15	0.15	0.04	0.10
6244-06	0.055	NEGATIVE	0.12	0.12	0.04	0.10
6244-07	0.055	NEGATIVE	0.11	0.11	0.06	0.12
6244-08	0.40	NEGATIVE	0.17	0.17	0.07	0.11
6244-09	0.085	NEGATIVE	0.19	0.19	0.09	0.10
6244-10	0.092	NEGATIVE	0.11	0.11	0.01	0.11
6244-11	0.077	P24	0.16	0.16	0.04	0.12
6244-12	0.099	P24	0.52	0.52	0.06	0.11
6244-13	0.121	P24	7.11	7.11	0.08	0.12
6244-14	0.158	P24	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED
6244-15	0.232	P24	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**  
ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)