



**HIV Seroconversion Panel Donor No. 65404
Part Number: HIV9013**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	CHIRON HIV-1 bDNA TEST DATE: 8/20/1998	ABBOTT HIV-1 p24 Ag TEST DATE: 10/3/1998	ABBOTT HIV-1 p24 Ag NT% TEST DATE: 9/3/1998	ABBOTT AxSYM HIV AB ½ EIA TEST DATE: 9/16/1998	ABBOTT HIV AB ½ EIA rDNA TEST DATE: UNK	ABBOTT MUREX HIV Ab ½ + O 3 RD Generation TEST DATE: UNK	ORTHO VITROS ECi ANTI-HIV ½ TEST DATE: 3/18/2015
9013-01	11/05/97	<50	0.09	NEGATIVE	0.32	0.342	0.459	0.15
9013-02	11/12/97	<50	0.06	NEGATIVE	0.31	0.171	0.439	0.15
9013-03	11/14/97	<50	0.09	NEGATIVE	0.31	0.171	0.420	0.15
9013-04	11/19/97	<50	0.08	NEGATIVE	0.29	0.171	0.449	0.15
9013-05	11/23/97	58	0.09	NEGATIVE	0.32	0.171	0.452	0.15
9013-06	11/28/97	56350	1.56	NEGATIVE	0.30	0.171	0.475	0.15
9013-07	11/30/97	185800	7.84	119.00	0.30	0.171	0.495	0.16

PANEL MEMBER	ABBOTT PRISM ANTI-HIV ½ TEST DATE: 8/18/1998	BIOMERIEUX ORGANON TEKNIKA ANTI-HIV-1 TEST DATE: 8/26/1998	CAMBRIDGE BIOTECH HIV-1 WESTERN BLOT TEST DATE: UNK	ABBOTT PRISM HIV Ag/Ab COMBO 4 th Generation TEST DATE: UNK	ABBOTT MUREX HIV Ag/Ab COMBO 4 th Generation TEST DATE: UNK	ABBOTT HIV Ag/Ab COMBO ARCHITECT 4 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 th Generation TEST DATE: UNK
9013-01	0.24	0.53	INDETERMINATE	0.06	0.255	0.149	0.06
9013-02	0.24	0.58	INDETERMINATE	0.06	0.235	0.119	0.10
9013-03	0.24	0.52	INDETERMINATE	0.07	0.230	0.106	0.15
9013-04	0.23	0.62	INDETERMINATE	0.07	0.260	0.126	0.10
9013-05	0.25	0.58	INDETERMINATE	0.07	0.280	0.092	0.13
9013-06	0.23	0.50	INDETERMINATE	0.44	0.735	0.489	1.44
9013-07	0.24	1.08	INDETERMINATE	2.07	2.480	2.13	5.00

PANEL MEMBER	BIO-RAD BioPlex 2200 HIV-1 Ag 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 th Generation TEST DATE: UNK	BIOMERIEUX ORGANON VIRNOSTIKA HIV UNI-FORM II PLUS 0 TEST DATE: 9/2/1998	BIOTEST ANTI-HIV ½ TEST DATE: 9/24/1998	DADE BEHRING ENZYGNOST ANTI-HIV ½ PLUS TEST DATE: 8/31/1998	GENETICS SYSTEMS HIV ½ EIA TEST DATE: 8/25/1998	ORTHO ANTI-HIV ½ ENHANCED ELISA TEST DATE: 9/15/1998
9013-01	0.06	0.02	0.40	0.026	0.01	0.121	0.01
9013-02	0.10	0.06	0.38	0.021	0.03	0.125	0.02
9013-03	0.15	0.06	0.38	0.023	0.03	0.134	0.02
9013-04	0.08	0.06	0.39	0.048	0.01	0.080	0.03
9013-05	0.13	0.05	0.39	0.010	0.02	0.105	0.02
9013-06	1.44	0.04	0.42	0.012	0.01	0.089	0.03
9013-07	5.00	0.09	0.43	0.030	0.02	0.118	0.02

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9013 Rev07
Effective Date: 09/20/2023