



**HIV Seroconversion Panel Donor No. 65522**  
**Part Number: HIV9014**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

| PANEL MEMBER | DATE OF DRAW | CHIRON HIV-1 bDNA<br>TEST DATE: 8/20/1998 | ABBOTT HIV-1 p24 Ag<br>TEST DATE: 8/27/1998 | ABBOTT HIV-1 p24 Ag NT%<br>TEST DATE: 9/3/1998 | COULTER HIV-1 p24 Ag<br>TEST DATE: 10/30/1998 | ABBOTT AxSYM HIV AB ½ EIA<br>TEST DATE: 9/16/1998 | ABBOTT HIV AB ½ EIA rDNA<br>TEST DATE: 8/26/1998 | ABBOTT MUREX HIV Ab ½ + O 3 <sup>RD</sup> Generation<br>TEST DATE: 8/20/1998 |
|--------------|--------------|---|---|--|---|---|--|--|
| 9014-01      | 11/13/97     | 25680                                     | 0.49  | NEGATIVE                                       | 1.15  | 0.36  | 0.171  | 0.36   |
| 9014-02      | 11/15/97     | 5614                                      | 0.37  | NEGATIVE                                       | 0.48  | 0.36  | 0.256  | 0.41   |
| 9014-03      | 11/23/97     | 281                                       | 0.26  | NEGATIVE                                       | 0.34  | 1.86  | 3.932  | 6.09   |
| 9014-04      | 11/25/97     | <50                                       | 0.35  | NEGATIVE                                       | 0.38  | 1.49  | 3.162  | 7.50   |
| 9014-05      | 12/07/97     | <50                                       | 0.36  | NEGATIVE                                       | 0.36  | 4.15  | 4.530  | 7.98   |
| 9014-06      | 12/12/97     | <50                                       | 0.34  | NEGATIVE                                       | 0.35  | 5.57  | 5.300  | 7.18   |
| 9014-07      | 12/14/97     | <50                                       | 0.42  | NEGATIVE                                       | 0.35  | 6.53  | 6.325  | 7.10   |

| PANEL MEMBER | ORTHO VITROS ECi ANTI-HIV ½<br>TEST DATE: 3/18/2015 | ABBOTT PRISM ANTI-HIV ½<br>TEST DATE: 8/18/1998 | BIOMERIEUX ORGANON TEKNIKA ANTI-HIV-1<br>TEST DATE: UNK | BIOMERIEUX ORGANON VIRNOSTIKA HIV UNI-FORM II PLUS 0<br>TEST DATE: 9/2/1998 | BIO-RAD MULTISPOT HIV 1 RAPID TEST<br>TEST DATE: UNK | BIO-RAD MULTISPOT HIV 2 RAPID TEST<br>TEST DATE: UNK | BIOTEST ANTI-HIV ½<br>TEST DATE: 9/24/1998 |
|--------------|---|---|---|---|--|--|--|
| 9014-01      | 0.24  | 0.28  | 0.75  | 0.41  | NEGATIVE   | NEGATIVE   | 0.038                                      |
| 9014-02      | 0.65  | 0.25  | 0.62  | 0.42  | NEGATIVE   | NEGATIVE   | 0.046                                      |
| 9014-03      | NOT TESTED  | 2.67  | 1.45  | 1.55  | NEGATIVE   | NEGATIVE   | 3.010                                      |
| 9014-04      | 11.3  | 2.92  | 1.00  | 1.53  | POSITIVE   | NEGATIVE   | 4.133                                      |
| 9014-05      | 17.5  | 20.99   | 3.00  | 9.08  | POSITIVE   | NEGATIVE   | 5.742                                      |
| 9014-06      | 20.2  | 24.76   | 2.83  | 9.75  | POSITIVE   | NEGATIVE   | 4.225                                      |
| 9014-07      | 20.2  | 27.99   | 2.67  | 10.19   | POSITIVE   | NEGATIVE   | 5.370                                      |

| PANEL MEMBER | DADE BEHRING ENZYGNOST ANTI-HIV ½ PLUS<br>TEST DATE: 8/31/1998 | GENETICS SYSTEMS HIV ½ EIA<br>TEST DATE: 8/25/1998 | ORTHO ANTI-HIV ½ ENHANCED ELISA<br>TEST DATE: 9/15/1998 | TRINITY BIOTECH UNIGOLD RECOMBIG EN HIV-1<br>TEST DATE: UNK | ORAQUICK ADVANCE HIV ½ RAPID Ab<br>TEST DATE: UNK | CAMBRIDGE BIOTECH HIV-1 WESTERN BLOT<br>TEST DATE: UNK |
|--------------|--|--|---|---|---|--|
| 9014-01      | 0.05   | 0.147  | 0.04  | NEGATIVE  | NEGATIVE  | P24  |
| 9014-02      | 0.04   | 0.166  | 0.20  | NEGATIVE  | NEGATIVE  | P24  |
| 9014-03      | 2.19   | 0.591  | 5.52  | NEGATIVE  | NEGATIVE  | INDETERMINATE  |
| 9014-04      | 1.79   | 0.792  | 5.70  | NEGATIVE  | NEGATIVE  | P17 P24 P160   |
| 9014-05      | 3.62   | 2.655  | 8.89  | NEGATIVE  | POSITIVE  | P17 P24 P66  |
| 9014-06      | 4.09   | 3.760  | 9.84  | NEGATIVE  | POSITIVE  | P17 P24 P66  |
| 9014-07      | 4.64   | 3.508  | 9.68  | NEGATIVE  | POSITIVE  | P17 P24 P66  |

**Notes on Data:**

- This panel is for **Research Use Only. Not for use in diagnostic procedures.**
- Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
- All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
- No preservatives have been added.
- All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**  
 ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)



**HIV Seroconversion Panel Donor No. 65522  
Part Number: HIV9014**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

| PANEL MEMBER | ABBOTT PRISM HIV Ag/Ab COMBO 4 <sup>th</sup> Generation<br>TEST DATE: UNK | ABBOTT MUREX HIV Ag/Ab COMBO 4 <sup>th</sup> Generation<br>TEST DATE: UNK | ABBOTT HIV Ag/Ab COMBO ARCHITECT 4 <sup>th</sup> Generation<br>TEST DATE: UNK | BIO-RAD BioPlex 2200 HIV Ag-Ab 5 <sup>th</sup> Generation<br>TEST DATE: UNK | BIO-RAD BioPlex 2200 HIV-1 Ag 5 <sup>th</sup> Generation<br>TEST DATE: UNK | BIO-RAD BioPlex 2200 HIV-1 Ab 5 <sup>th</sup> Generation<br>TEST DATE: UNK |
|--------------|---|---|---|---|--|--|
| 9014-01      | 0.58  | 0.782   | 1.48  | NOT TESTED  | NOT TESTED   | NOT TESTED   |
| 9014-02      | 0.30  | 0.406   | 0.35  | 0.41  | 0.41   | 0.05   |
| 9014-03      | 1.64  | 4.365   | 18.04   | NOT TESTED  | NOT TESTED   | NOT TESTED   |
| 9014-04      | 1.71  | 4.178   | 40.72   | 11.16   | 0.12   | 11.16  |
| 9014-05      | 1.79  | 17.162  | 54.69   | 32.77   | 0.05   | 32.77  |
| 9014-06      | 2.35  | 17.898  | 37.46   | 13.55   | 0.05   | 13.55  |
| 9014-07      | 2.55  | 18.127  | 28.65   | 9.81  | 0.09   | 9.81   |

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**  
ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)