



**HIV Seroconversion Panel Donor No. 65661
Part Number: HIV9015**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	CHIRON HIV-1 bDNA	ABBOTT HIV-1 p24 Ag	ABBOTT HIV-1 p24 Ag NT%	COULTER HIV-1 p24 Ag	ABBOTT AxSYM HIV AB ½ EIA	ABBOTT HIV AB ½ EIA rDNA	ABBOTT MUREX HIV Ab ½ + O 3 RD Generation
		TEST DATE: 8/20/1998	TEST DATE: 7/21/1998	TEST DATE: 9/3/1998	TEST DATE: 8/26/1998	TEST DATE: 9/16/1998	TEST DATE: 6/29/1998	TEST DATE: UNK
9015-01	11/29/97	<50	0.317	NEGATIVE	0.56	0.37	0.256	0.367
9015-02	12/03/97	<50	0.317	NEGATIVE	0.56	0.33	0.256	0.322
9015-03	12/06/97	<50	0.317	NEGATIVE	0.72	0.32	0.171	0.269
9015-04	12/10/97	<50	0.317	NEGATIVE	0.56	0.30	0.171	0.272
9015-05	12/17/97	<50	0.317	NEGATIVE	0.56	0.32	0.171	0.269
9015-06	12/20/97	898	0.317	NEGATIVE	0.56	0.35	0.171	0.265
9015-07	12/27/97	>500000	>31.746	MAX ABS.	>24.0	0.32	0.171	0.304
9015-08	12/29/97	>500000	>31.746	MAX ABS.	20.8	0.40	0.171	0.350
9015-09	01/03/98	>500000	>31.746	MAX ABS.	22.4	2.10	7.863	7.375
9015-10	01/05/98	>500000	>31.746	MAX ABS.	22.4	4.21	11.538	11.223

PANEL MEMBER	ORTHO VITROS ECi ANTI-HIV 1+2	ABBOTT PRISM ANTI-HIV ½	TRINITY BIOTECH UNIGOLD RECOMBIGNEN HIV-1	ORAQUICK ADVANCE HIV ½ RAPID Ab	CAMBRIDGE BIOTECH HIV-1 WESTERN BLOT	BIOMERIEUX ORGANON TEKNIKA ANTI-HIV-1	BIOMERIEUX ORGANON VIRNOSTIKA HIV UNI-FORM II PLUS 0
	TEST DATE: 3/18/2015	TEST DATE: 8/18/1998	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 8/26/1998	TEST DATE: 9/2/1998
9015-01	NOT TESTED	0.25	NEGATIVE	NEGATIVE	NEGATIVE	0.53	0.42
9015-02	NOT TESTED	0.26	NEGATIVE	NEGATIVE	NEGATIVE	0.52	0.40
9015-03	0.17	0.22	NEGATIVE	NEGATIVE	NEGATIVE	0.55	0.39
9015-04	NOT TESTED	0.24	NEGATIVE	NEGATIVE	NEGATIVE	0.53	0.40
9015-05	0.17	0.23	NEGATIVE	NEGATIVE	NEGATIVE	0.58	0.39
9015-06	0.16	0.23	NEGATIVE	NEGATIVE	NEGATIVE	0.48	0.39
9015-07	NOT TESTED	0.24	NEGATIVE	NEGATIVE	NEGATIVE	0.53	0.42
9015-08	0.43	0.25	NEGATIVE	NEGATIVE	NEGATIVE	0.52	0.47
9015-09	13.1	1.82	NEGATIVE	NEGATIVE	P24 P160 (VF)	0.60	2.60
9015-10	NOT TESTED	4.80	POSITIVE	POSITIVE	P24-P160	0.92	2.88

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com



**HIV Seroconversion Panel Donor No. 65661
Part Number: HIV9015**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	BIOTEST ANTI-HIV ½ TEST DATE: 9/22/1998	DADE BEHRING ENZYGNOST ANTI-HIV ½ PLUS TEST DATE: 8/31/1998	GENETICS SYSTEMS HIV ½ EIA TEST DATE: UNK	ORTHO ANTI-HIV ½ ENHANCED ELISA TEST DATE: 9/15/1998	BIO-RAD MULTISPOT HIV 1 RAPID TEST TEST DATE: UNK	BIO-RAD MULTISPOT HIV 2 RAPID TEST TEST DATE: UNK
9015-01	0.003	0.04	0.133	0.02	NEGATIVE	NEGATIVE
9015-02	0.025	0.04	0.272	0.01	NEGATIVE	NEGATIVE
9015-03	0.012	0.02	0.340	0.01	NEGATIVE	NEGATIVE
9015-04	0.010	0.03	0.282	0.02	NEGATIVE	NEGATIVE
9015-05	0.030	0.03	0.223	0.00	NEGATIVE	NEGATIVE
9015-06	0.031	0.02	0.133	0.01	NEGATIVE	NEGATIVE
9015-07	0.039	0.03	0.194	0.02	NEGATIVE	NEGATIVE
9015-08	0.036	0.06	1.126	0.34	NEGATIVE	NEGATIVE
9015-09	2.630	5.23	9.709	9.54	NEGATIVE	NEGATIVE
9015-10	4.910	6.95	9.709	10.88	POSITIVE	NEGATIVE

PANEL MEMBER	ABBOTT PRISM HIV Ag/Ab COMBO 4 th Generation TEST DATE: UNK	ABBOTT MUREX HIV-1 Ab/Ag COMBO 4 th Generation TEST DATE: UNK	ABBOTT HIV Ag/Ab COMBO ARCHITECT 4 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 th Generation TEST DATE: UNK
9015-01	0.07	0.289	0.14	NOT TESTED	NOT TESTED	NOT TESTED
9015-02	0.06	0.218	0.13	0.14	0.14	0.06
9015-03	0.08	0.213	0.11	0.13	0.12	0.10
9015-04	0.11	0.223	0.09	0.10	0.10	0.04
9015-05	0.08	0.234	0.12	0.12	0.10	0.07
9015-06	0.11	0.249	0.17	0.10	0.10	0.04
9015-07	219.96	18.096	185.57	NOT TESTED	NOT TESTED	NOT TESTED
9015-08	204.66	18.264	99.41	>200	>200	0.10
9015-09	65.47	18.208	17.91	>200	>200	0.83
9015-10	43.29	18.503	8.56	NOT TESTED	NOT TESTED	NOT TESTED

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com