



**HIV Seroconversion Panel Donor No. 65790
Part Number: HIV9016**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

| PANEL MEMBER | DATE OF DRAW | CHIRON HIV-1 bDNA | ABBOTT HIV-1 p24 Ag | ABBOTT HIV-1 p24 Ag NT% | COULTER HIV-1 p24 Ag | ABBOTT AxSYM HIV AB ½ EIA | ABBOTT HIV AB ½ EIA rDNA | ABBOTT MUREX HIV Ab ½ + O 3 RD Generation | ORTHO VITROS ECi ANTI-HIV 1/2 |
|--------------|--------------|----------------------|-----------------------|-------------------------|----------------------|---------------------------|--------------------------|--|-------------------------------|
| | | TEST DATE: 8/20/1998 | TEST DATE: 10/16/1998 | TEST DATE: 9/3/1998 | TEST DATE: 10/3/1998 | TEST DATE: 9/16/1998 | TEST DATE: 8/27/1998 | TEST DATE: UNK | TEST DATE: 3/18/2015 |
| 9016-01 | 12/11/97 | <50 | 0.301 | NEGATIVE | 0.05 | 0.34 | 0.256 | 0.293 | 0.16 |
| 9016-02 | 12/13/97 | <50 | 0.260 | NEGATIVE | 0.79 | 0.35 | 0.171 | 0.272 | 0.16 |
| 9016-03 | 12/18/97 | <50 | 0.260 | NEGATIVE | 0.05 | 0.35 | 0.256 | 0.283 | 0.16 |
| 9016-04 | 12/20/97 | <50 | 0.288 | NEGATIVE | 0.05 | 0.37 | 0.171 | 0.283 | 0.16 |
| 9016-05 | 12/26/97 | <50 | 0.247 | NEGATIVE | 0.07 | 0.34 | 0.171 | 0.283 | 0.16 |
| 9016-06 | 12/29/97 | <50 | 0.247 | NEGATIVE | 0.08 | 0.34 | 0.171 | 0.297 | 0.16 |
| 9016-07 | 01/03/98 | <50 | 0.260 | NEGATIVE | 0.05 | 0.36 | 0.171 | 0.318 | 0.16 |
| 9016-08 | 01/07/98 | 7473 | 0.288 | NEGATIVE | 0.32 | 0.35 | 0.171 | 0.339 | 0.16 |
| 9016-09 | 01/10/98 | 69010 | 0.767 | 86.60 | 3.36 | 0.33 | 0.256 | 0.297 | 0.16 |
| 9016-10 | 01/14/98 | 286400 | 3.014 | 99.00 | 13.22 | 0.38 | 0.171 | 0.300 | 0.27 |

| PANEL MEMBER | ORTHO ECi ANTI-HIV ½ | ABBOTT PRISM ANTI-HIV ½ | ZEPTOMETRIX HIV p24 Ag (pg/mL) | CAMBRIDGE BIOTECH HIV-1 WESTERN BLOT | BIOMERIEUX ORGANON TEKNIKA ANTI-HIV-1 | BIOMERIEUX ORGANON VIRNOSTIKA HIV UNI-FORM II PLUS 0 | BIOTEST ANTI-HIV ½ | DADE BEHRING ENZYGNOST ANTI-HIV ½ PLUS |
|--------------|----------------------|-------------------------|--------------------------------|--------------------------------------|---------------------------------------|--|----------------------|--|
| | TEST DATE: UNK | TEST DATE: 8/18/1998 | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: 8/26/1998 | TEST DATE: 9/2/1998 | TEST DATE: 4/22/1998 | TEST DATE: 8/31/1998 |
| 9016-01 | 0.06 | 0.25 | <3.0 | NEGATIVE | 0.50 | 0.39 | 0.010 | 0.04 |
| 9016-02 | 0.04 | 0.22 | <3.0 | NEGATIVE | 0.53 | 0.40 | 0.000 | 0.02 |
| 9016-03 | 0.06 | 0.23 | <3.0 | NEGATIVE | 0.48 | 0.40 | 0.015 | 0.03 |
| 9016-04 | 0.06 | 0.21 | <3.0 | NEGATIVE | 0.47 | 0.39 | 0.003 | 0.03 |
| 9016-05 | 0.06 | 0.21 | <3.0 | NEGATIVE | 0.47 | 0.42 | 0.007 | 0.02 |
| 9016-06 | 0.06 | 0.22 | <3.0 | NEGATIVE | 0.52 | 0.42 | 0.002 | 0.02 |
| 9016-07 | 0.06 | 0.22 | <3.0 | NEGATIVE | 0.50 | 0.44 | 0.021 | 0.02 |
| 9016-08 | 0.06 | 0.22 | <3.0 | NEGATIVE | 0.50 | 0.41 | 0.007 | 0.14 |
| 9016-09 | 0.06 | 0.22 | <3.0 | NEGATIVE | 0.52 | 0.43 | 0.000 | 0.03 |
| 9016-10 | 0.06 | 0.28 | 19.2 | NEGATIVE | 0.50 | 0.40 | 0.002 | 0.02 |

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com



**HIV Seroconversion Panel Donor No. 65790
Part Number: HIV9016**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

| PANEL MEMBER | GENETICS SYSTEMS HIV 1/2 EIA TEST DATE: 8/25/1998 | ORTHO ANTI-HIV 1/2 ENHANCED ELISA TEST DATE: 9/15/1998 | ABBOTT PRISM HIV Ag/Ab COMBO 4 th Generation TEST DATE: UNK | ABBOTT MUREX HIV-1 Ab/Ag COMBO 4 th Generation TEST DATE: UNK | ABBOTT ARCHITECT HIV Ag/Ab COMBO 4 th Generation TEST DATE: UNK | BIO-RAD BioPlex 2200 HIV Ag-Ab 5 th Generation TEST DATE: UNK | BIO-RAD BioPlex 2200 HIV-1 Ag 5 th Generation TEST DATE: UNK | BIO-RAD BioPlex 2200 HIV-1 Ab 5 th Generation TEST DATE: UNK |
|--------------|---|---|--|--|--|---|--|--|
| 9016-01 | 0.102 | 0.02 | 0.06 | 0.330 | 0.13 | 0.08 | 0.08 | 0.06 |
| 9016-02 | 0.080 | 0.01 | 0.06 | 0.249 | 0.07 | 0.07 | 0.07 | 0.02 |
| 9016-03 | 0.083 | 0.00 | 0.10 | 0.249 | 0.11 | 0.07 | 0.07 | 0.02 |
| 9016-04 | 0.083 | 0.01 | 0.07 | 0.279 | 0.14 | 0.10 | 0.10 | 0.02 |
| 9016-05 | 0.077 | 0.01 | 0.07 | 0.264 | 0.11 | 0.12 | 0.12 | 0.06 |
| 9016-06 | 0.089 | 0.02 | 0.06 | 0.279 | 0.15 | 0.09 | 0.09 | 0.06 |
| 9016-07 | 0.096 | 0.02 | 0.09 | 0.315 | 0.10 | 0.08 | 0.08 | 0.03 |
| 9016-08 | 0.086 | 0.01 | 0.20 | 0.381 | 0.21 | 0.67 | 0.67 | 0.05 |
| 9016-09 | 0.083 | 0.02 | 1.11 | 1.345 | 1.41 | 5.54 | 5.54 | 0.04 |
| 9016-10 | 0.099 | 0.02 | 5.94 | 6.340 | 4.45 | 34.36 | 34.36 | 0.05 |

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com