



**HIV Seroconversion Panel Donor No. 67706
Part Number: HIV9023**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	CHIRON HIV-1 bDNA	ABBOTT HIV-1 p24 Ag	COULTER HIV-1 p24 Ag	ROCHE ELECSYS HIV p24Ag	ABBOTT AxSYM HIV AB ½ EIA	ABBOTT HIV AB ½ EIA rDNA	ABBOTT HIV AB ½ EIA rDNA 3 rd Generation	ABBOTT MUREX HIV Ab ½ + O 3 rd Generation
		TEST DATE: 10/26/1999	TEST DATE: 6/16/1999	TEST DATE: 6/18/1999	TEST DATE: 6/21/1999	TEST DATE: 7/14/1999	TEST DATE: 5/13/1999	TEST DATE: 7/12/1999	TEST DATE: UNK
9023-01	04/15/98	<50	0.37	0.602	0.28	0.28	0.157	0.26	0.366
9023-02	04/17/98	<50	0.39	0.482	0.33	0.20	0.182	0.31	0.325
9023-03	04/22/98	<50	0.38	0.482	0.23	0.22	0.165	0.27	0.328
9023-04	04/24/98	<50	0.41	0.482	0.26	0.21	0.149	0.26	0.375
9023-05	04/29/98	<50	0.41	0.482	0.24	0.20	0.190	0.29	0.378
9023-06	05/01/98	<50	0.45	0.361	0.91	0.20	0.182	0.28	0.472
9023-07	05/13/98	<50	0.38	0.361	0.24	0.21	0.207	0.31	0.388
9023-08	05/15/98	<50	0.46	0.361	0.28	0.21	0.182	0.28	0.372
9023-09	05/20/98	<50	0.29	0.241	0.26	0.23	0.165	0.27	0.325
9023-10	05/22/98	<50	0.44	0.482	0.24	0.22	0.174	0.32	0.309
9023-11	05/27/98	<50	0.40	0.361	0.28	0.22	0.190	0.33	0.316
9023-12	05/29/98	<50	0.43	0.482	0.29	0.21	0.182	0.24	0.359
9023-13	06/09/98	<50	0.40	0.602	0.24	0.20	0.174	0.27	0.428
9023-14	06/11/98	<50	0.50	0.361	0.31	0.22	0.182	0.27	0.522
9023-15	06/16/98	<50	0.45	0.361	0.29	0.21	0.165	0.28	0.403
9023-16	06/18/98	<50	0.39	0.361	0.36	0.22	0.165	0.26	0.356
9023-17	06/23/98	<50	0.40	0.361	0.33	0.20	0.174	0.28	0.331
9023-18	06/25/98	<50	0.46	0.361	0.33	0.20	0.182	0.31	0.319
9023-19	06/30/98	7183	0.45	0.602	0.50	0.22	0.149	0.26	0.309
9023-20	07/02/98	62566	0.73	2.048	2.16	0.21	0.149	0.28	0.381
9023-21	07/07/98	>500000	7.98	38.554	56.14	0.20	0.182	0.24	0.400
9023-22	07/09/98	>500000	11.88	40.964	83.63	0.21	0.198	0.34	0.488

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9023 Rev06
Effective Date: 03/13/2023



**HIV Seroconversion Panel Donor No. 67706
Part Number: HIV9023**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT HIV AB ½ EIA rDNA IMx TEST DATE: 7/19/1999	ABBOTT MUREX ANTI-HIV ½ ICE + O TEST DATE: 7/21/1999	ABBOTT PRISM ANTI-HIV ½ TEST DATE: 7/15/1999	BIOMERIEUX ORGANON VIRNOSTIKA HIV-1 MICROELISA SYSTEM TEST DATE: 6/18/1999	BIOMERIEUX ORGANON VIRNOSTIKA HIV UNI-FORM II PLUS 0 TEST DATE: 6/9/1999	BIO-RAD NOVAPATH HIV-1 IMMUNOBLOT TEST DATE: UNK	BIOTEST ANTI-HIV ½ TEST DATE: 6/9/1999	BIOTEST ANTI-HIV-1 TETRA ELISA TEST DATE: 6/8/1999
9023-01	0.47	0.20	0.31	0.267	0.46	NEGATIVE	0.06	0.27
9023-02	0.38	0.17	0.23	0.267	0.58	NEGATIVE	0.05	0.25
9023-03	0.31	0.17	0.22	0.297	0.51	NEGATIVE	0.05	0.27
9023-04	0.35	0.17	0.24	0.267	0.48	NEGATIVE	0.03	0.23
9023-05	0.39	0.16	0.24	0.267	0.45	NEGATIVE	0.05	0.25
9023-06	0.46	0.17	0.22	0.267	0.42	NEGATIVE	0.04	0.23
9023-07	0.36	0.15	0.22	0.237	0.44	NEGATIVE	0.05	0.33
9023-08	0.39	0.16	0.25	0.237	0.44	NEGATIVE	0.05	0.24
9023-09	0.36	0.29	0.19	0.237	0.46	NEGATIVE	0.04	0.28
9023-10	0.33	0.17	0.31	0.267	0.45	NEGATIVE	0.05	0.30
9023-11	0.42	0.15	0.22	0.267	0.51	NEGATIVE	0.08	0.24
9023-12	0.38	0.16	0.45	0.267	0.46	NEGATIVE	0.05	0.25
9023-13	0.41	0.17	0.24	0.267	0.48	NEGATIVE	0.07	0.26
9023-14	0.34	0.15	0.25	0.267	0.50	NEGATIVE	0.06	0.25
9023-15	0.34	0.15	0.24	0.297	0.47	NEGATIVE	0.06	0.26
9023-16	0.33	0.16	0.31	0.267	0.51	NEGATIVE	0.05	0.31
9023-17	0.37	0.19	0.25	0.267	0.49	NEGATIVE	0.05	0.25
9023-18	0.36	0.18	0.23	0.237	0.51	NEGATIVE	0.05	0.29
9023-19	0.31	0.16	0.23	0.237	0.51	NEGATIVE	0.07	0.27
9023-20	0.34	0.14	0.24	0.237	0.50	NEGATIVE	0.07	0.30
9023-21	0.32	0.14	0.24	0.237	0.41	NEGATIVE	0.06	0.24
9023-22	0.34	0.24	0.26	0.237	0.45	NEGATIVE	0.06	0.55

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9023 Rev06
Effective Date: 03/13/2023



**HIV Seroconversion Panel Donor No. 67706
Part Number: HIV9023**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DADE BEHRING ENZYGNOST ANTI-HIV ½ PLUS TEST DATE: 6/8/1999	GENETICS SYSTEMS HIV ½ PEPTIDE EIA TEST DATE: 6/28/1999	GENETICS SYSTEMS rLAV EIA TEST DATE: 11/15/1999	ORTHO ANTI-HIV ½ ENHANCED ELISA TEST DATE: 6/29/1999	ROCHE ENZYMUM ANTI-HIV-COMBI TEST DATE: 7/14/1999	SANOFI GENSCREEN ANTI-HIV ½ TEST DATE: 6/21/1999	ABBOTT PRISIM HIV Ag/Ab COMBO 4 TH Generation TEST DATE: UNK	ABBOTT MUREX HIV-1 Ab/Ag COMBO 4 TH Generation TEST DATE: UNK
9023-01	0.13	0.211	0.056	0.05	0.09	0.06	0.11	0.213
9023-02	0.12	0.211	0.074	0.05	0.08	0.05	0.10	0.213
9023-03	0.10	0.211	0.065	0.05	0.09	0.05	0.11	0.223
9023-04	0.03	0.141	0.065	0.05	0.08	0.05	0.07	0.228
9023-05	0.10	0.317	0.053	0.03	0.09	0.08	0.08	0.279
9023-06	0.12	0.352	0.059	0.04	0.08	0.07	0.09	0.279
9023-07	0.13	0.458	0.062	0.04	0.09	0.06	0.05	0.467
9023-08	0.15	0.176	0.056	0.05	0.09	0.04	0.08	0.274
9023-09	0.15	0.106	0.053	0.05	0.08	0.05	0.06	0.218
9023-10	0.12	0.141	0.065	0.05	0.09	0.06	0.06	0.223
9023-11	0.11	0.106	0.062	0.04	0.08	0.04	0.06	0.213
9023-12	0.11	0.176	0.074	0.04	0.08	0.04	0.11	0.218
9023-13	0.11	0.070	0.068	0.05	0.08	0.06	0.06	0.239
9023-14	0.11	0.106	0.071	0.05	0.08	0.07	0.07	0.254
9023-15	0.14	0.211	0.077	0.06	0.10	0.06	0.09	0.340
9023-16	0.12	0.141	0.062	0.05	0.10	0.05	0.09	0.218
9023-17	0.13	0.106	0.062	0.04	0.10	0.07	0.08	0.223
9023-18	0.13	0.106	0.065	0.06	0.09	0.05	0.07	0.223
9023-19	0.04	0.141	0.065	0.06	0.10	0.07	0.23	0.305
9023-20	0.12	0.106	0.074	0.04	0.11	0.03	0.82	1.056
9023-21	0.12	0.176	0.059	0.09	1.83	0.13	26.41	17.523
9023-22	0.13	0.211	0.056	1.18	2.91	6.75	37.38	18.914

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9023 Rev06
Effective Date: 03/13/2023



HIV Seroconversion Panel Donor No. 67706
Part Number: HIV9023

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT ARCHITECT HIV Ag/Ab COMBO 4 TH Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 th Generation TEST DATE: UNK	ORTHO VITROS ECi ANTI-HIV 1/2 TEST DATE: 8/30/2018
9023-01	0.11	0.09	0.09	0.04	0.10
9023-02	0.11	0.08	0.08	0.04	0.09
9023-03	0.13	0.10	0.10	0.04	0.09
9023-04	0.12	0.07	0.07	0.01	0.10
9023-05	0.14	0.04	0.03	0.01	0.10
9023-06	0.12	0.15	0.15	0.05	0.10
9023-07	0.13	0.04	0.04	0.01	0.11
9023-08	0.15	0.07	0.06	0.04	0.10
9023-09	0.13	0.11	0.11	0.04	0.10
9023-10	0.13	0.06	0.06	0.04	0.10
9023-11	0.13	0.08	0.08	0.02	0.10
9023-12	0.12	0.08	0.08	0.06	0.10
9023-13	0.13	0.05	0.05	0.01	0.10
9023-14	0.35	0.15	0.15	0.05	0.11
9023-15	0.12	0.08	0.08	0.03	0.10
9023-16	0.12	0.11	0.11	0.07	0.10
9023-17	0.11	0.03	0.02	0.02	0.10
9023-18	0.13	0.11	0.11	0.09	0.10
9023-19	0.30	0.21	0.21	0.05	0.09
9023-20	1.94	3.73	3.73	0.02	0.10
9023-21	58.51	98.83	98.83	0.05	0.12
9023-22	87.79	125.10	125.10	0.08	0.84

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9023 Rev06
 Effective Date: 03/13/2023