



**HIV Seroconversion Panel Donor No. 64469**  
**Part Number: HIV9024**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	CHIRON HIV-1 bDNA	ABBOTT HIV-1 p24 Ag	COULTER HIV-1 p24 Ag	ROCHE ELECSYS HIV p24 Ag	ABBOTT AxSYM HIV AB ½ EIA	ABBOTT HIV AB ½ EIA rDNA	ABBOTT HIV AB ½ EIA rDNA 3 <sup>rd</sup> Generation	ABBOTT MUREX HIV Ab ½ + O 3 <sup>rd</sup> Generation
		TEST DATE: 10/26/1999	TEST DATE: 5/11/1999	TEST DATE: 6/18/1999	TEST DATE: 6/10/1999	TEST DATE: 7/14/1999	TEST DATE: 5/11/1999	TEST DATE: 6/14/1999	TEST DATE: UNK
9024-01	06/06/97	<50	0.203	0.241	0.50	0.30	0.129	0.22	0.314
9024-02	06/08/97	<50	0.237	0.241	0.49	0.23	0.121	0.20	0.290
9024-03	06/13/97	<50	0.220	0.241	0.47	0.23	0.112	0.22	0.290
9024-04	06/15/97	<50	0.169	0.361	0.47	0.22	0.112	0.20	0.290
9024-05	06/20/97	<50	0.169	0.145	0.49	0.23	0.138	0.19	0.332
9024-06	06/22/97	<50	0.254	0.241	0.47	0.23	0.147	0.21	0.357
9024-07	06/27/97	<50	0.254	0.361	0.43	0.21	0.138	0.19	0.428
9024-08	07/11/97	<50	0.254	0.361	0.39	0.27	0.103	0.21	0.371
9024-09	07/13/97	<50	0.254	0.361	0.44	0.22	0.121	0.25	0.367
9024-10	07/21/97	<50	0.254	0.361	0.37	0.20	0.095	0.21	0.325
9024-11	07/25/97	12840	0.288	0.602	0.38	0.23	0.138	0.18	0.322
9024-12	07/29/97	>500000	6.610	12.410	37.10	0.24	0.138	0.21	0.343

PANEL MEMBER	BIOMERIEUX ORGANON VIRNOSTIKA HIV UNI-FORM II PLUS 0	BIO-RAD NOVAPATH HIV-1 IMMUNOBLOT	BIOTEST ANTI-HIV ½	ABBOTT HIV AB ½ EIA rDNA IMx	ABBOTT MUREX SUDS ANTI-HIV-1	ABBOTT PRISM ANTI-HIV ½	BIOMERIEUX ORGANON VIRNOSTIKA HIV-1 MICROELISA SYSTEM
	TEST DATE: 7/13/1999	TEST DATE: UNK	TEST DATE: 6/9/1999	TEST DATE: 12/7/1999	TEST DATE: 7/21/1999	TEST DATE: 7/15/1999	TEST DATE: 6/18/1999
9024-01	0.45	NEGATIVE	0.02	0.31	0.14	0.28	0.237
9024-02	0.43	NEGATIVE	0.04	0.30	0.15	0.22	0.267
9024-03	0.43	NEGATIVE	0.03	0.30	0.14	0.28	0.208
9024-04	0.47	NEGATIVE	0.04	0.30	0.18	0.24	0.208
9024-05	0.41	NEGATIVE	0.04	0.31	0.15	0.24	0.208
9024-06	1.84	NEGATIVE	0.02	0.31	0.14	0.24	0.237
9024-07	0.48	NEGATIVE	0.02	0.29	0.14	0.34	0.237
9024-08	0.41	NEGATIVE	0.05	0.29	0.14	0.23	0.237
9024-09	0.27	NEGATIVE	0.05	0.33	0.15	0.23	0.237
9024-10	0.38	NEGATIVE	0.04	0.32	0.16	0.26	0.237
9024-11	0.42	NEGATIVE	0.06	1.07	0.15	0.28	0.237
9024-12	0.49	NEGATIVE	0.05	0.36	0.23	0.33	0.237

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**  
 ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525

For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)



**HIV Seroconversion Panel Donor No. 64469  
Part Number: HIV9024**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	BIOTEST ANTI-HIV-1 TETRA ELISA  TEST DATE: 6/8/1999	DADE BEHRING ENZYGNOST ANTI-HIV ½ PLUS  TEST DATE: 6/8/1999	GENETICS SYSTEMS HIV ½ PEPTIDE EIA  TEST DATE: 6/28/1999	GENETICS SYSTEMS rLAV EIA  TEST DATE: 11/15/1999	ORTHO ANTI-HIV ½ ENHANCED ELISA  TEST DATE: 6/21/1999	ROCHE ENZYMMUM ANTI-HIV-COMBI  TEST DATE: 7/14/1999	SANOFI GENSCREEN ANTI-HIV ½  TEST DATE: 6/21/1999	ABBOTT PRISIM HIV Ag/Ab COMBO 4 <sup>TH</sup> Generation  TEST DATE: UNK
9024-01	0.15	0.12	0.053	NOT TESTED	0.04	0.31	0.13	0.08
9024-02	0.18	0.11	0.049	NOT TESTED	0.04	0.33	0.14	0.13
9024-03	0.16	0.16	0.046	NOT TESTED	0.05	0.33	0.12	0.10
9024-04	0.15	0.14	0.046	NOT TESTED	0.05	0.31	0.12	0.12
9024-05	0.17	0.14	0.056	NOT TESTED	0.05	0.29	0.12	0.15
9024-06	0.15	0.14	0.060	NOT TESTED	0.04	0.32	0.09	0.09
9024-07	0.15	0.12	0.056	NOT TESTED	0.04	0.29	0.11	0.10
9024-08	0.17	0.13	0.042	NOT TESTED	0.05	0.29	0.10	0.08
9024-09	0.15	0.12	0.049	NOT TESTED	0.05	0.30	0.11	0.06
9024-10	0.17	0.13	0.039	NOT TESTED	0.06	0.27	0.10	0.13
9024-11	0.16	0.14	0.056	0.065	0.04	0.30	0.14	0.25
9024-12	0.17	0.15	0.056	0.056	0.05	1.61	0.12	25.94

PANEL MEMBER	ABBOTT MUREX HIV-1 Ab/Ag COMBO 4 <sup>TH</sup> Generation  TEST DATE: UNK	ABBOTT ARCHITECT HIV Ag/Ab COMBO 4 <sup>TH</sup> Generation  TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 <sup>th</sup> Generation  TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 <sup>th</sup> Generation  TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 <sup>th</sup> Generation  TEST DATE: UNK	ORTHO VITROS ECi ANTI-HIV ½  TEST DATE: 8/30/2018
9024-01	0.249	0.08	0.03	0.03	0.01	0.11
9024-02	0.244	0.08	0.04	0.04	0.01	0.11
9024-03	0.234	0.08	0.08	0.08	0.06	0.11
9024-04	0.249	0.10	0.09	0.09	0.01	0.11
9024-05	0.244	0.10	0.05	0.05	0.00	0.12
9024-06	0.249	0.11	0.10	0.10	0.07	0.11
9024-07	0.249	0.08	0.05	0.05	0.01	0.11
9024-08	0.249	0.06	0.10	0.10	0.03	0.10
9024-09	0.274	0.07	0.06	0.06	0.02	0.11
9024-10	0.305	0.07	0.03	0.02	0.00	0.11
9024-11	0.406	0.38	0.84	0.84	0.02	0.11
9024-12	13.868	30.84	94.57	94.57	0.01	0.11

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**  
ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525

For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)