



**HIV Seroconversion Panel Donor No. 68205
Part Number: HIV9026**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	CHIRON HIV-1 bDNA TEST DATE: 10/26/1999	ABBOTT HIV-1 p24 Ag TEST DATE: 6/16/1999	COULTER HIV-1 p24 Ag TEST DATE: 8/3/1999	ROCHE ELECSYS HIV p24Ag TEST DATE: 6/10/1999	ABBOTT AxSYM HIV AB ½ EIA TEST DATE: 7/14/1999	ABBOTT HIV AB ½ EIA rDNA TEST DATE: 8/4/1999	ABBOTT HIV AB ½ EIA rDNA 3 rd Generation TEST DATE: 6/14/1999
9026-01	07/21/98	<50	0.41	0.096	0.27	0.27	0.082	0.19
9026-02	07/26/98	<50	0.41	0.322	0.33	0.22	0.073	0.18
9026-03	08/03/98	<50	0.44	0.165	0.29	0.23	0.073	0.18
9026-04	08/05/98	<50	0.41	0.174	0.28	0.28	0.091	0.18
9026-05	08/16/98	<50	0.44	0.139	0.24	0.27	0.073	0.20
9026-06	08/21/98	<50	0.44	0.148	0.31	0.23	0.064	0.18
9026-07	09/03/98	211703	1.44	5.504	0.38	1.06	1.136	1.32

PANEL MEMBER	ABBOTT MUREX HIV Ab ½ + O 3 rd Generation TEST DATE: UNK	ABBOTT HIV AB ½ EIA rDNA IMx TEST DATE: 12/7/1999	ABBOTT MUREX SUDS ANTI-HIV-1 TEST DATE: 7/21/1999	ORTHO VITROS ECi ANTI-HIV 1/2 TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HIV 1/2 TEST DATE: 7/21/2015	ABBOTT PRISM ANTI-HIV ½ TEST DATE: 7/15/1999	BIOMERIEUX ORGANON VIRNOSTIKA HIV-1 MICROELISA SYSTEM TEST DATE: 7/27/1999	BIOMERIEUX ORGANON VIRNOSTIKA HIV UNI-FORM II PLUS 0 TEST DATE: 6/9/1999	BIO-RAD NOVAPATH HIV-1 IMMUNOBLOT TEST DATE: UNK
9026-01	0.353	0.36	0.22	0.03	0.11	0.26	0.491	0.37	P24
9026-02	0.388	0.33	0.22	0.07	0.11	0.33	0.470	0.38	P24
9026-03	0.409	0.34	0.20	0.06	0.11	0.21	0.470	0.39	P24
9026-04	0.453	0.36	0.24	0.06	0.11	0.25	0.449	0.39	P24
9026-05	0.347	0.34	0.23	0.07	0.11	0.24	0.449	0.00	P24
9026-06	0.300	0.36	0.26	0.06	0.11	0.26	0.470	0.41	P24
9026-07	5.256	2.90	8.03	14.7	12.3	0.64	0.491	1.23	P24

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525

For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com



HIV Seroconversion Panel Donor No. 68205
Part Number: HIV9026

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	BIOTEST ANTI-HIV ½ TEST DATE: 6/9/1999	BIOTEST ANTI-HIV-1 TETRA ELISA TEST DATE: 6/8/1999	DADE BEHRING ENZYGNOST ANTI-HIV ½ PLUS TEST DATE: 6/8/1999	GENETICS SYSTEMS HIV ½ PEPTIDE EIA TEST DATE: 7/29/1999	GENETICS SYSTEMS rLAV EIA TEST DATE: 11/15/1999	ORTHO ANTI-HIV ½ ENHANCED ELISA TEST DATE: 6/21/1999	ROCHE ENZYMMUM ANTI-HIV-COMBI TEST DATE: 7/14/1999
9026-01	0.06	0.20	0.12	0.156	0.068	0.05	0.30
9026-02	0.05	0.25	0.15	0.156	0.071	0.05	0.30
9026-03	0.04	0.13	0.11	0.188	0.065	0.06	0.30
9026-04	0.03	0.21	0.14	0.156	0.059	0.13	0.30
9026-05	0.04	0.27	0.11	0.125	0.056	0.05	0.48
9026-06	0.05	0.15	0.12	0.094	0.059	0.07	0.48
9026-07	0.33	3.27	0.77	0.188	0.059	10.26	3.89

PANEL MEMBER	SANOVI GENSCREEN ANTI-HIV ½ TEST DATE: 6/21/1999	ABBOTT PRISIM HIV Ag/Ab COMBO 4 TH Generation TEST DATE: UNK	ABBOTT MUREX HIV-1 Ab/Ag COMBO 4 TH Generation TEST DATE: UNK	ABBOTT ARCHITECT HIV Ag/Ab COMBO 4 TH Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 th Generation TEST DATE: UNK
9026-01	0.05	0.08	0.254	0.18	0.16	0.16	0.06
9026-02	0.08	0.06	0.218	0.10	0.18	0.18	0.06
9026-03	0.10	0.07	0.218	0.13	0.13	0.13	0.06
9026-04	0.11	0.07	0.228	0.09	0.18	0.18	0.07
9026-05	0.49	0.08	0.244	0.11	0.14	0.14	0.05
9026-06	0.06	0.09	0.239	0.08	0.12	0.12	0.07
9026-07	2.84	2.83	4.848	4.20	13.12	13.12	0.10

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525

For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com