



**HIV Seroconversion Panel Donor No. 62216**  
**Product Code: HIV9075**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	BLEED DATE	GEN- PROBE PROCLEIX HIV-1 HCV ASSAY	ROCHE COBAS AMPLICOR HIV-1 MONITOR	ABBOTT HIV-1 p24 Ag	COULTER HIV-1 p24 Ag	ZEPTOMETRIX HIV p24 Ag (pg/mL)	ABBOTT HIV Ab ½ EIA rDNA 3 <sup>rd</sup> Generation	ABBOTT MUREX HIV Ab ½ + O 3 <sup>rd</sup> Generation
9075-01	08-22-96	NEG	NOT DETECTED	NOT TESTED	0.37	<3.0	0.144	0.424
9075-02	08-24-96	NEG	NOT DETECTED	NOT TESTED	0.247	<3.0	0.153	0.341
9075-03	09-13-96	<b>POS</b>	<b>&gt;750,000</b>	<b>2.30</b>	<b>5.222</b>	<b>22.2</b>	0.161	0.359
9075-04	09-23-96	<b>POS</b>	<b>&gt;750,000</b>	<b>3.33</b>	<b>1.889</b>	<b>30.2</b>	<b>16.949</b>	<b>10.430</b>
9075-05	09-25-96	<b>POS</b>	<b>&gt;750,000</b>	NOT TESTED	<b>1.667</b>	<b>69.7</b>	<b>16.949</b>	<b>10.601</b>

PANEL MEMBER	ORTHO ECi Anti-HIV 1/2	BIOMERIEUX VIRNOSTIKA ANTI HIV-1	BIO-RAD ANTI HIV-1 rLAV	GENETICS SYSTEMS HIV ½ + O	MEDMIRA ANTI-HIV 1 RAPID TEST	TRINITY BIO-TECH UNI GOLD RECOMBIGEN	BIO-RAD MULTISPOT HIV 1 RAPID TEST	BIO-RAD MULTISPOT HIV 2 RAPID TEST	ORAQUICK ADVANCE HIV ½ RAPID Ab
9075-01	0.14	0.455	0.041	0.102	NEG	NEG	NEG	NEG	NEG
9075-02	0.14	0.370	0.044	0.239	NEG	NEG	NEG	NEG	NEG
9075-03	0.18	0.415	0.041	0.167	NEG	NEG	NEG	NEG	NEG
9075-04	<b>38.8</b>	0.944	<b>3.764</b>	<b>10.239</b>	<b>POS</b>	<b>POS</b>	<b>POS</b>	NEG	<b>POS</b>
9075-05	<b>33.5</b>	<b>1.086</b>	<b>4.159</b>	<b>10.239</b>	<b>POS</b>	<b>POS</b>	<b>POS</b>	NEG	<b>POS</b>

PANEL MEMBER	BIO-RAD HIV-1 WESTERN BLOT	ABBOTT PRISM HIV Ag/Ab COMBO 4 <sup>th</sup> Generation	ABBOTT MUREX HIV-1 Ab/Ag COMBO 4 <sup>th</sup> Generation	ABBOTT HIV Ag/Ab COMBO ARCHITECT 4 <sup>th</sup> Generation	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 <sup>th</sup> Generation	BIO-RAD BioPlex 2200 HIV-1 Ag 5 <sup>th</sup> Generation	BIO-RAD BioPlex 2200 HIV-1 Ab 5 <sup>th</sup> Generation
9075-01	NEG	0.24	0.324	0.063	0.12	0.12	0.03
9075-02	NEG	0.09	0.275	0.071	0.06	0.04	0.05
9075-03	NEG	<b>3.42</b>	<b>5.049</b>	<b>7.27</b>	<b>28.08</b>	<b>28.08</b>	0.06
9075-04	<b>gp160, p24+/-, p18 +/-</b>	<b>10.28</b>	<b>16.887</b>	<b>31.86</b>	<b>34.38</b>	<b>34.38</b>	<b>27.61</b>
9075-05	<b>gp160, p55+/-, p40 +/-, p24, p18 +/-</b>	<b>17.14</b>	<b>17.162</b>	<b>51.01</b>	<b>33.38</b>	<b>5.02</b>	<b>33.38</b>

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)