



**HIV Seroconversion Panel Donor No. 75018  
Part Number: HIV9077**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	GEN-PROBE PROCLEIX HIV-1 /HCV ASSAY  TEST DATE: UNK	BIO-RAD HIV-1 WESTERN BLOT  TEST DATE: UNK	ZEPTOMETRIX HIV p24 Ag (pg/mL)  TEST DATE: UNK	ABBOTT HIV AB 1/2 EIA rDNA  TEST DATE: 6/3/2002	ABBOTT MUREX HIV Ab 1/2 + O 3 <sup>RD</sup> Generation  TEST DATE: UNK
9077-01	10/20/99	NEGATIVE	NOT TESTED	<3.0	0.153	0.511
9077-02	10/25/99	NEGATIVE	NOT TESTED	<3.0	0.144	0.461
9077-03	10/27/99	NEGATIVE	NOT TESTED	<3.0	0.162	0.393
9077-04	11/02/99	NEGATIVE	NOT TESTED	<3.0	0.162	0.396
9077-05	11/04/99	NEGATIVE	NOT TESTED	<3.0	0.144	0.368
9077-06	11/09/99	NEGATIVE	NOT TESTED	<3.0	0.144	0.427
9077-07	11/11/99	NEGATIVE	NOT TESTED	<3.0	0.135	0.412
9077-08	11/16/99	NEGATIVE	NOT TESTED	<3.0	0.126	0.511
9077-09	11/18/99	NEGATIVE	NOT TESTED	<3.0	0.135	0.545
9077-10	11/23/99	NEGATIVE	NOT TESTED	<3.0	0.162	0.458
9077-11	11/30/99	POSITIVE	NOT TESTED	<3.0	0.153	0.412
9077-12	12/04/99	POSITIVE	NEGATIVE	6.3	0.198	0.415
9077-13	12/09/99	POSITIVE	NEGATIVE	105.5	0.135	0.560
9077-14	12/11/99	POSITIVE	NEGATIVE	>125	0.432	1.003
9077-15	12/16/99	POSITIVE	GP160,P55,P40,P24	29.3	8.793	10.582
9077-16	12/18/99	POSITIVE	GP160,P55,P40,P24	4.8	6.721	10.573
9077-17	12/23/99	POSITIVE	GP160,P55,P40,P24	<3.0	9.514	10.477
9077-18	12/26/99	POSITIVE	GP160,P55,P40,P24	<3.0	10.225	10.551
9077-19	12/30/99	POSITIVE	GP160,P65+/-,P55,P40,P24,P18+/-	<3.0	11.919	10.362
9077-20	01/02/00	POSITIVE	GP160,P65,P55,P40,P31+/-,P24,P18+/-	<3.0	12.730	10.384
9077-21	01/06/00	POSITIVE	GP160,P65,P55,P40,P31,P24,P18+/-	<3.0	13.649	10.421
9077-22	01/09/00	POSITIVE	GP160,P65,P55,P40,P31,P24,P18	<3.0	13.757	10.511
9077-23	01/13/00	POSITIVE	GP160,P65,P55,P40,P31,P24,P18	<3.0	15.009	NOT TESTED
9077-24	01/16/00	POSITIVE	GP160,P65,P55,GP41+/-,P40,P31,P24,P18	<3.0	15.306	10.517
9077-25	01/20/00	POSITIVE	GP160,P65,P55,GP41+/-,P40,P31,P24,P18	<3.0	17.820	10.885
9077-26	01/23/00	POSITIVE	GP160,P65,P55,GP41+/-,P40,P31,P24,P18	<3.0	18.018	10.833
9077-27	01/27/00	POSITIVE	GP160,GP120+/-,P65,P55,GP41+/-,P40,P31,P24,P18	<3.0	18.018	10.721
9077-28	01/30/00	POSITIVE	GP160,GP120+/-,P65,P55,GP41+/-,P40,P31,P24,P18	<3.0	18.018	10.771
9077-29	02/03/00	POSITIVE	GP160,GP120+/-,P65,P55,GP41+/-,P40,P31,P24,P18	<3.0	18.018	10.780

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)



**HIV Seroconversion Panel Donor No. 75018  
Part Number: HIV9077**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	BIOMERIEUX VIRONOSTIKA ANTI HIV-1  TEST DATE: UNK	BIO-RAD ANTI HIV-1 rLAV  TEST DATE: 6/3/2002	GENETICS SYSTEMS HIV 1/2 + O  TEST DATE: UNK	TRINITY BIO-TECH UNI GOLD RECOMBIGN HIV-1  TEST DATE: UNK	BIO-RAD MULTISPOT HIV 1 RAPID TEST  TEST DATE: UNK	BIO-RAD MULTISPOT HIV 2 RAPID TEST  TEST DATE: UNK	MEDMIRA ANTI-HIV-1 RAPID TEST  TEST DATE: UNK	ORAQUICK ADVANCE HIV 1/2 RAPID Ab  TEST DATE: UNK
9077-01	0.421	0.055	0.334	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9077-02	0.348	0.066	0.133	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9077-03	0.308	0.033	0.191	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9077-04	0.368	0.063	0.133	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9077-05	0.325	0.055	0.154	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9077-06	0.375	0.041	0.106	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9077-07	0.461	0.052	0.106	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9077-08	0.700	0.055	0.109	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9077-09	0.453	0.052	0.304	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9077-10	0.338	0.066	0.119	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9077-11	0.287	0.066	0.191	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9077-12	0.408	0.081	0.119	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9077-13	0.392	0.066	0.526	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9077-14	0.405	0.081	5.280	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	POSITIVE
9077-15	1.598	0.771	10.239	POSITIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE
9077-16	2.833	1.373	10.239	POSITIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE
9077-17	2.107	1.642	10.239	POSITIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE
9077-18	2.437	1.716	10.239	NEGATIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE
9077-19	1.927	1.875	10.239	NEGATIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE
9077-20	2.081	2.148	10.239	POSITIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE
9077-21	1.912	2.336	10.239	POSITIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE
9077-22	2.014	2.708	10.239	POSITIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE
9077-23	2.477	2.993	10.239	POSITIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE
9077-24	4.335	3.421	10.239	POSITIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE
9077-25	2.844	3.779	10.239	POSITIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE
9077-26	2.793	3.996	10.239	POSITIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE
9077-27	3.273	3.343	10.239	POSITIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE
9077-28	3.225	4.033	10.239	POSITIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE
9077-29	3.606	5.413	10.239	POSITIVE	POSITIVE	NEGATIVE	POSITIVE	POSITIVE

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)



**HIV Seroconversion Panel Donor No. 75018  
Part Number: HIV9077**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT PRISM HIV Ag/Ab COMBO 4 <sup>TH</sup> Generation  TEST DATE: UNK	ABBOTT MUREX MICROPLATE HIV Ag/Ab COMBO 4 <sup>TH</sup> Generation  TEST DATE: UNK	ABBOTT HIV Ag/Ab COMBO ARCHITECT 4 <sup>TH</sup> Generation  TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 <sup>th</sup> Generation  TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 <sup>th</sup> Generation  TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 <sup>th</sup> Generation  TEST DATE: UNK	ORTHO VITROS ECi ANTI-HIV 1/2  TEST DATE: 8/4/2015
9077-01	0.17	0.240	0.09	0.10	0.10	0.01	0.13
9077-02	0.06	0.250	0.28	0.07	0.07	0.03	0.13
9077-03	0.10	0.240	0.09	0.05	0.03	0.04	0.14
9077-04	0.06	0.245	0.19	0.13	0.13	0.02	0.13
9077-05	0.08	0.240	0.13	0.06	0.04	0.06	0.14
9077-06	0.08	0.255	0.15	0.08	0.08	0.03	0.13
9077-07	0.07	0.270	0.10	0.05	0.04	0.03	0.12
9077-08	0.07	0.270	0.22	0.07	0.07	0.06	0.15
9077-09	0.11	0.245	0.12	0.11	0.11	0.03	0.13
9077-10	0.10	0.235	0.11	0.06	0.04	0.05	0.13
9077-11	0.08	0.294	0.15	0.17	0.17	0.03	0.13
9077-12	2.19	2.245	2.83	10.00	10.00	0.04	0.14
9077-13	29.22	16.279	36.86	147.15	147.15	0.07	0.59
9077-14	37.81	16.980	42.80	166.91	166.91	0.86	2.43
9077-15	14.75	13.819	20.42	24.73	24.49	24.73	25.3
9077-16	5.66	9.078	17.99	25.27	4.02	25.27	29.4
9077-17	2.92	11.520	11.14	19.58	0.62	19.58	25.8
9077-18	2.71	13.583	8.45	17.18	0.49	17.18	21.9
9077-19	2.39	15.966	5.99	11.93	0.29	11.93	22.4
9077-20	2.82	16.471	5.38	10.73	0.19	10.73	25.2
9077-21	3.10	16.966	5.07	9.14	0.12	9.14	26.9
9077-22	3.47	16.863	5.64	9.54	0.07	9.54	32.2
9077-23	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED
9077-24	5.09	16.877	9.54	15.32	0.12	15.32	31.8
9077-25	5.92	18.206	15.37	25.19	0.09	25.19	36.3
9077-26	6.47	18.652	15.81	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED
9077-27	7.24	17.980	23.08	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED
9077-28	8.15	17.760	33.61	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED
9077-29	9.47	17.873	51.72	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**  
ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)