



**HIV Seroconversion Panel Donor No. 63215
Part Number: HIV9081**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	GEN- PROBE PROCLEIX HIV-1 HCV ASSAY	ROCHE COBAS AMPLICOR HIV-1 MONITOR	ABBOTT HIV-1 p24 Ag	COULTER HIV-1 p24 Ag	ZEPTOMETRIX HIV p24 Ag (pg/mL)	ABBOTT HIV Ab ½ EIA rDNA	ABBOTT MUREX HIV Ab ½ + O 3 rd Generation
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
9081-01	01/17/97	NEGATIVE	NEGATIVE	NOT TESTED	NOT TESTED	<3.0	0.153	0.392
9081-02	02/10/97	POSITIVE	>750,000	NOT TESTED	NOT TESTED	61.1	16.949	11.781
9081-03	02/12/97	POSITIVE	422,000	15.87	9.42	11.8	16.949	11.885
9081-04	02/19/97	POSITIVE	269,000	12.61	2.72	<3.0	12.856	11.990

PANEL MEMBER	ORTHO VITROS ECi ANTI-HIV 1/2	BIOMERIEUX VIRNOSTIKA ANTI-HIV-1	BIO-RAD ANTI-HIV-1 rLAV	GENETICS SYSTEMS HIV ½ + O	MEDMIRA ANTI-HIV 1 RAPID TEST	TRINITY BIO-TECH UNI GOLD RECOMBIGEN	BIO-RAD MULTISPOT HIV 1 RAPID TEST	BIO-RAD MULTISPOT HIV 2 RAPID TEST	ORAQUICK ADVANCE HIV ½ RAPID Ab
	TEST DATE: 8/6/2015	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
9081-01	0.19	0.474	0.111	0.205	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9081-02	56.8	0.442	1.535	10.239	POSITIVE	POSITIVE	POSITIVE	NEGATIVE	POSITIVE
9081-03	64.6	0.474	3.576	10.239	POSITIVE	POSITIVE	POSITIVE	NEGATIVE	POSITIVE
9081-04	33.9	2.305	5.819	10.239	POSITIVE	POSITIVE	POSITIVE	NEGATIVE	POSITIVE

PANEL MEMBER	BIO-RAD HIV-1 WESTERN BLOT	ABBOTT PRISM HIV Ag/Ab COMBO 4 th Generation	ABBOTT MUREX MICROPLATE HIV Ab/Ag COMBO 4 th Generation	ABBOTT HIV Ag/Ab COMBO ARCHITECT 4 th Generation	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 th Generation	BIO-RAD BioPlex 2200 HIV-1 Ag 5 th Generation	BIO-RAD BioPlex 2200 HIV-1 Ab 5 th Generation
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
9081-01	NEGATIVE	0.12	0.247	0.13	0.07	0.06	0.04
9081-02	GP160+/-,P55,P40,P24	11.96	16.884	44.24	2.07	2.07	0.38
9081-03	GP160,P55,P40,P24	6.50	16.672	51.06	31.07	13.37	31.07
9081-04	GP160,GP120+/-, P55,P40,P24	3.18	16.394	53.31	74.24	1.56	74.24

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com