



**HIV Seroconversion Panel Donor No. 77840  
Part Number: HIV9082**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

| PANEL MEMBER | DATE OF DRAW | ROCHE COBAS AMPLICOR HIV-1 MONITOR<br>TEST DATE: UNK | ZEPTOMETRIX HIV p24 Ag (pg/mL)<br>TEST DATE: UNK | ORTHO VITROS Eci ANTI-HIV 1/2<br>TEST DATE: 8/6/2015 | ABBOTT HIV AB 1/2 EIA rDNA<br>TEST DATE: UNK | GENETICS SYSTEMS HIV 1/2 + O<br>TEST DATE: UNK | MEDMIRA ANTI-HIV-1 RAPID TEST<br>TEST DATE: UNK | ORAQUICK ADVANCE HIV 1/2 RAPID Ab<br>TEST DATE: UNK |
|--------------|--------------|------------------------------------------------------|--------------------------------------------------|------------------------------------------------------|----------------------------------------------|------------------------------------------------|-------------------------------------------------|-----------------------------------------------------|
| 9082-01      | 01/30/00     | NEGATIVE                                             | 0.8                                              | 0.13                                                 | 0.3                                          | 0.1                                            | NOT TESTED                                      | NOT TESTED                                          |
| 9082-02      | 02/03/00     | NEGATIVE                                             | 0.94                                             | 0.14                                                 | 0.1                                          | 0.1                                            | NOT TESTED                                      | NOT TESTED                                          |
| 9082-03      | 02/12/00     | 52800                                                | 1.94                                             | 0.16                                                 | 0.1                                          | 0.1                                            | NEGATIVE                                        | NEGATIVE                                            |
| 9082-04      | 02/19/00     | 604000                                               | 2.24                                             | 18.2                                                 | 3.6                                          | 0.5                                            | NEGATIVE                                        | NEGATIVE                                            |

| PANEL MEMBER | TRINITY BIO-TECH UNI GOLD RECOMBIGEN HIV-1<br>TEST DATE: UNK | BIO-RAD MULTISPOT HIV 1 RAPID TEST<br>TEST DATE: UNK | CHEMBIO STAT PAK RAPID HIV 1/2<br>TEST DATE: UNK | CHEMBIO SURE CHECK RAPID HIV 1/2<br>TEST DATE: UNK | CAMBRIDGE BIOTECH HIV-1 WESTERN BLOT<br>TEST DATE: UNK | BIO-RAD BioPlex 2200 HIV Ag-Ab 5 <sup>th</sup> Generation<br>TEST DATE: UNK | BIO-RAD BioPlex 2200 HIV-1 Ag 5 <sup>th</sup> Generation<br>TEST DATE: UNK | BIO-RAD BioPlex 2200 HIV-1 Ab 5 <sup>th</sup> Generation<br>TEST DATE: UNK |
|--------------|--------------------------------------------------------------|------------------------------------------------------|--------------------------------------------------|----------------------------------------------------|--------------------------------------------------------|-----------------------------------------------------------------------------|----------------------------------------------------------------------------|----------------------------------------------------------------------------|
| 9082-01      | NOT TESTED                                                   | NOT TESTED                                           | NOT TESTED                                       | NOT TESTED                                         | NOT TESTED                                             | 0.06                                                                        | 0.06                                                                       | 0.01                                                                       |
| 9082-02      | NOT TESTED                                                   | NOT TESTED                                           | NOT TESTED                                       | NOT TESTED                                         | NOT TESTED                                             | 0.12                                                                        | 0.12                                                                       | 0.01                                                                       |
| 9082-03      | NEGATIVE                                                     | NEGATIVE                                             | NEGATIVE                                         | NEGATIVE                                           | NEGATIVE                                               | 6.77                                                                        | 6.77                                                                       | 0.01                                                                       |
| 9082-04      | NEGATIVE                                                     | NEGATIVE                                             | NEGATIVE                                         | NEGATIVE                                           | NEGATIVE                                               | >200                                                                        | >200                                                                       | 0.64                                                                       |

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**  
ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525

For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)