



**HIV Seroconversion Panel Donor No. 77600  
Part Number: HIV9084**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ORTHO VITROS ECI ANTI-HIV 1/2  TEST DATE: UNK	CHIRON HIV-1 RNA bDNA  TEST DATE: 10/18/2000	ROCHE COBAS AMPLICOR HIV-1 MONITOR  TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 <sup>th</sup> Generation  TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 <sup>th</sup> Generation  TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 <sup>th</sup> Generation  TEST DATE: UNK	ZEPTOMETRIX HIV p24 Ag (pg/mL)  TEST DATE: 8/23/2007	ABBOTT HIV AB 1/2 EIA rDNA  TEST DATE: UNK
9084-01	02/07/00	0.12	<50	NEGATIVE	0.13	0.13	0.07	0.5	0.15
9084-02	02/10/00	0.11	<50	NEGATIVE	0.13	0.13	0.08	0.4	0.14
9084-03	02/17/00	0.12	1077	1290	0.20	0.20	0.10	0.6	0.14
9084-04	03/27/00	30.6	29097	85400	32.31	0.80	32.31	0.6	14.3

PANEL MEMBER	GENETICS SYSTEMS rLAV EIA  TEST DATE: UNK	MEDMIRA ANTI-HIV-1 RAPID TEST  TEST DATE: UNK	ORAQUICK ADVANCE HIV 1/2 RAPID Ab  TEST DATE: UNK	TRINITY BIO-TECH UNI GOLD RECOMBIGEN HIV-1  TEST DATE: UNK	BIO-RAD MULTISPOT HIV 1 RAPID TEST  TEST DATE: UNK	CHEMBIO STAT PAK RAPID HIV 1/2  TEST DATE: UNK	CHEMBIO SURE CHECK RAPID HIV 1/2  TEST DATE: UNK	CAMBRIDGE BIOTECH HIV-1 WESTERN BLOT  TEST DATE: UNK
9084-01	0.1	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED
9084-02	0.1	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED
9084-03	0.1	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9084-04	5.5	POSITIVE	POSITIVE	POSITIVE	POSITIVE	POSITIVE	POSITIVE	POSITIVE

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**  
ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)