



**HIV Seroconversion Panel Donor No. 65376  
Part Number: HIV9089**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ORTHO VITROS Eci ANTI-HIV 1/2 TEST DATE: UNK	CHIRON HIV-1 RNA bDNA TEST DATE: UNK	ABBOTT HIV-1 p24 Ag TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 <sup>th</sup> Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 <sup>th</sup> Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 <sup>th</sup> Generation TEST DATE: UNK	ABBOTT HIV-1 p24 Ag NT% TEST DATE: UNK
9089-01	11/09/97	0.14	<50	0.28	0.06	0.06	0.00	NEGATIVE
9089-02	11/16/97	0.12	<50	0.35	0.06	0.06	0.02	NEGATIVE
9089-03	11/18/97	0.13	316	0.32	0.09	0.09	0.02	NEGATIVE
9089-04	11/25/97	1.02	146954	2.88	24.93	24.93	0.00	100.2
9089-05	11/29/97	29.1	59647	0.77	6.45	6.45	0.01	113.5
9089-06	12/03/97	37.8	42837	1.25	5.72	5.72	0.45	124

PANEL MEMBER	GENETICS SYSTEMS HIV ½ EIA TEST DATE: UNK	COULTER HIV-1 p24 Ag TEST DATE: UNK	INNOTEST ANTI-HIV 1/2½ TEST DATE: UNK	ABBOTT HIV AB ½ EIA rDNA IMx TEST DATE: UNK	ABBOTT AxSYM HIV AB ½ EIA TEST DATE: UNK	ABBOTT HIV AB ½ EIA rDNA TEST DATE: UNK	ABBOTT MUREX HIV AB ½ + O 3 <sup>rd</sup> Generation TEST DATE: UNK	ABBOTT HIV AB ½ EIA rDNA 3 <sup>rd</sup> Generation TEST DATE: UNK
9089-01	0.173	0.233	0.067	0.28	0.32	0.234	0.428	0.21
9089-02	0.211	0.233	0.043	0.30	0.35	0.211	0.386	0.19
9089-03	0.198	0.233	0.050	0.31	0.33	0.234	0.419	0.22
9089-04	0.201	8.023	0.063	0.35	0.33	0.313	0.590	0.17
9089-05	0.188	2.910	0.050	1.25	0.73	1.211	9.673	0.44
9089-06	0.355	2.910	2.286	4.69	2.35	4.422	10.354	2.33

PANEL MEMBER	ORTHO ANTI-HIV ½ ENHANCED ELISA TEST DATE: UNK	ABBOTT PRISM HIV Ag/Ab COMBO 4 <sup>th</sup> Generation TEST DATE: UNK	ABBOTT MUREX MICROPLATE HIV AG/AB COMBO 4 <sup>th</sup> Generation TEST DATE: UNK	ABBOTT HIV AG/AB COMBO ARCHITECT 4 <sup>th</sup> Generation TEST DATE: UNK	ABBOTT PRISM ANTI-HIV ½ TEST DATE: UNK	BIOTEST ANTI-HIV ½ TEST DATE: UNK
9089-01	0.10	0.09	0.281	0.11	0.34	0.06
9089-02	0.03	0.10	0.291	0.12	0.38	0.08
9089-03	0.02	0.17	0.305	0.11	0.37	0.09
9089-04	0.67	4.31	3.897	4.33	0.33	0.13
9089-05	9.99	2.09	4.103	1.01	0.43	1.14
9089-06	10.94	3.35	10.749	2.24	1.63	4.89

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)