



Zika Virus Seroconversion Panel Donor No. PMR1001029
Product Code: KZMC047
For Research Use Only. Not for Use in Diagnostic Procedures.

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and symptomatic throughout the collection period.

PANEL MEMBER	BLEED DATE	EUROIMMUN Zika IgM (S/Co)	EUROIMMUN Zika IgG (Result)	EUROIMMUN ChikV IgM (S/Co)	EUROIMMUN ChikV IgG (Result)	EUROIMMUN Dengue IgM (S/Co)	EUROIMMUN Dengue IgG (Result)
KZMC047-01	8/23/2016	3.00	19.9 (IND)	0.18	1.8	0.39	8.0
KZMC047-02	8/26/2016	2.68	23.0	0.15	1.6	0.33	6.8
KZMC047-03	8/30/2016	2.02	22.0 (IND)	0.14	1.7	0.29	7.2
KZMC047-04	9/2/2016	1.74	35.6	0.10	1.6	0.23	8.5
KZMC047-05	9/6/2016	1.10 (IND)	26.3	0.09	1.6	0.17	6.1
KZMC047-06	9/10/2016	0.76	22.0 (IND)	0.07	1.4	0.15	6.1
KZMC047-07	9/13/2016	0.66	29.9	0.08	1.6	0.15	7.4

PANEL MEMBER	BLEED DATE	EUROIMMUN WNV IgM (S/Co)	EUROIMMUN WNV IgG (Result)	RT-PCR	DIACHECK Anti-Zika IgG S/Co	DIACHECK Anti-Zika IgM S/Co
KZMC047-01	8/23/2016	0.34	1.5	Undetectable	3.832	2.963
KZMC047-02	8/26/2016	0.36	1.4	Undetectable	4.106	2.923
KZMC047-03	8/30/2016	0.34	1.6	Undetectable	4.644	2.286
KZMC047-04	9/2/2016	0.26	1.6	Undetectable	4.261	1.769
KZMC047-05	9/6/2016	0.25	1.9	Undetectable	4.598	1.382
KZMC047-06	9/10/2016	0.24	2.3	Undetectable	4.684	1.012
KZMC047-07	9/13/2016	0.26	2.5	Undetectable	4.024	0.609

Notes on Data: This panel is for investigational use only and the data is offered for informational purposes only. Data is generated at ZeptoMetrix Corporation and by our Lab Partners in the United States and Europe. We will endeavor to update our data sheets whenever new test data becomes available to us. All specimens collected in this longitudinal series are naturally occurring, undiluted, unprocessed samples collected from a single plasma donor in the United States. No Preservatives have been added.

All specimens tested for HBsAg and Anti-HCV and found to be negative by an FDA approved test.

Use Universal Precautions!

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com