

NATtrol™ HIV-1 Linearity Panel Strain: IIIB

PRODUCT DESCRIPTION:

NATtrol™ HIV-1 Linearity Panel (NATHIV1-LIN)* is formulated with purified, intact viral particles that have been chemically modified to render them non-infectious and refrigerator stable. NATHIV1-LIN contains 6 x 0.25 mL vials of HIV-1 NATtrol™ at concentrations listed in Table 1. The panels members are provided in a proprietary matrix.

*Pat.: <http://www.zeptometrix.com/patent-information/>

INTENDED USE:

- NATtrol™ HIV-1 Linearity Panel is designed to evaluate the performance of nucleic acid tests for determination of the presence of HIV-1 RNA. NATHIV1-LIN can also be used for validation of clinical assays, development of diagnostic tests and training of laboratory personnel.

WARNINGS AND PRECAUTIONS:

- NATtrol™ inactivation was carried out the HIV-1 stock used to formulate this product. The inactivation was verified in a standard microbiological growth protocol.
- This product contains inactivated microorganisms and materials of human and animal origin. Safe practices suggest that the controls be considered potentially infectious and to use Universal Precautions when handling.
- Refer to CDC guidelines and local regulations for handling and disposal.
- The matrix used in the manufacture of this product is treated with 0.09% sodium azide. It was manufactured from Human Serum Albumin that has been tested and found to be non-reactive at the donor level for HIV-1/HIV-2 Antibody, HBsAg and HCV Antibody by FDA licensed donor screening test methods. All materials are also tested for HIV-1 and HCV by FDA approved Nucleic Acid Test (NAT) methods.
- Heat inactivated Fetal Bovine Serum used in the manufacture of this product meets applicable USDA requirements for abattoir sourced animals, traceability and country of origin. The materials were collected at USDA licensed establishments or legally imported from countries recognized by the USDA as negligible or controlled for risk for Bovine Spongiform Encephalopathy (BSE) and other exotic disease agents. Donor animals were inspected ante and post mortem at the abattoir as required by the USDA.
- Do not use past the expiration date on the label.
- To avoid cross-contamination, use separate pipette tips for all materials.

RECOMMENDED STORAGE:

- NATtrol™ HIV-1 Linearity Panel should be stored at 2-8°C.

INSTRUCTIONS FOR USE:

- Mix vial vigorously for at least 5 secs.
- Process according to manufacturer’s instructions for sample to result assays.
- Extract nucleic acid prior to use in downstream assays that are not sample to result.

LIMITATION:

- FOR RESEARCH USE ONLY. NOT FOR USE IN DIAGNOSTIC PROCEDURES**
- Quality control materials should be used in accordance with local, state, federal, and accreditation requirements.
- This product is not intended to replace the manufacturer’s controls provided with the assay.

EXPECTED RESULTS:








- Each laboratory must evaluate the product and establish their own acceptance criteria.
- The table shown below is for informational purposes only.

Table 1: NATHIV1-LIN PANEL MEMBERS

Panel Member	Target Concentration (IU/mL) ¹
HIV1 1E6	1,000,000
HIV1 1E5	100,000
HIV1 1E4	10,000
HIV1 1E3	1,000
HIV1 4E2	400
Negative	Negative

¹Quantitation based on in-house real time PCR assay targeting the gag gene region.

PINATHIV1-LIN
Revision: 11
Effective Date: 02/24/2022

	Catalog Number		Temperature Limitation
	Batch Code		Expiration Date
	For Research Use Only		Biological Risk
	Manufacturer		

PCA# 22-094
Page 1 of 1